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Ziegelei 1

D-72336 Balingen

E-Mail: info@kern-sohn.com

Tel.: +49-[0]7433-9933-0

Fax: +49-[0]7433-9933-149

Internet: www.kern-sohn.com

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KERN MSF 200

Version 1.9 2025-07

Operating instructions Mechanical height measuring rod

Contents

1	Technical data	12
1.1	Tolerances body height measuring rod	12
2	Declaration of conformity	13
3	Explanation of the graphic symbols	13
4	Basic Information (General).....	14
4.1	Specific function.....	14
4.1.1	Indication	14
4.1.2	Contraindication.....	14
4.2	Proper use	14
4.3	Non-intended product use / contraindications	14
4.4	Warranty	15
4.5	Monitoring of Test Resources.....	15
4.6	Plausibility check	15
4.7	Reporting serious incidents	15
5	Basic Safety Precautions	16
5.1	Personnel training	16
5.2	Preventing contamination	16
5.3	Preparation for use.....	16
5.4	Safety Instructions	16
6	Unpacking/installing.....	17
6.1	Testing upon acceptance	17
6.2	Packaging / return transport	17
6.3	Scope of delivery	17
6.4	Checking the scale	17
6.5	Assembly.....	18
7	Body height measurement.....	19
8	Cleaning, Maintenance, Disposal	20
8.1	Cleaning / disinfecting	20
8.2	Sterilisation	20
8.3	Servicing, maintenance	20
8.4	Disposal.....	20

1 Technical data

KERN	MSF 200N
Measuring range	60 – 205 cm
Readability	1 mm
Tolerance (measuring range 60 – 100 cm)	± 5 mm
Tolerance (measuring range 100 – 205 cm)	± 10 mm
Dimensions (D x H x W) mm	60 x 290 x 930 mm
Temperature range	+10°C ... +40°C
Storage temperature	- 20°C ... + 60°C
Net weight	approx. 700 g
Admitted as medical product as per (EU) 2017/745	Category I with measuring function

1.1 Tolerances body height measuring rod

For assembly to the balances MTS-M, MPS-PM and MPB-P or for wall mounting.

Measured value (cm)	Tolerance (cm)
60 - 100	± 0.5
100-205	± 1.0

2 Declaration of conformity

The current EC/EU Conformity declaration can be found online in:

www.kern-sohn.com/ce

3 Explanation of the graphic symbols

The height meters fulfill the requirements of the regulation for medical products (EU) 2017/745

Unique product identification

Is a medical device

SN WD 20xxx

Designation of the serial number of every device, applied at the device and on the packaging.

Number here as example

2022-06

Identification of the manufacturing date of the medical product.

Year and month here as example

„Attention observe accompanying document“, or
„Follow operating instructions“

**Kern & Sohn GmbH
D-72336 Balingen
Ziegelei 1**

Manufacturer identification of the medical product including address

Information

4 Basic Information (General)

	This height measuring rod is provided to determine the body height in the medical practice and for medical control, examination and treatment according to medical product regulation (EU) 2017/745.
---	--

4.1 Specific function

4.1.1 Indication

Medical Height Rod:

Indication:

- Determination of body height in the medical field

Use:

- To determine a person's height, the person stands under the head marker. This is slowly lowered to the top of the head. The size in cm is read on the scale.

4.1.2 Contraindication

No contraindication known.

4.2 Proper use

This body height measuring rod is designed for determining the height of a person whilst standing, in medical treatment rooms. This height measuring rod is suitable to detect, prevent and survey diseases.

Ensure that the body height measuring rod may only come into contact only with uninjured skin.

The body height measuring rod should be checked for correct condition prior to each utilization by a person familiar with proper operation of the measuring rod.

When using balances with body height measuring rod, ensure that the top flap is turned downwards immediately after use in order to avoid risk of injury.

4.3 Non-intended product use / contraindications

	Non-intended use of the optional body height measuring rod • The body height measuring rod may only be assembled as specified in the operating instructions. • The structure of the body height measuring rod may not be modified. This may result in incorrect measuring results, safety-related defects as well as destruction. • The body height measuring rod may only be used according to the described conditions. Other areas of use must be released by KERN in writing. For more details please see the user manuals of the body height measuring rod.
---	---

4.4 Warranty

Warranty claims shall be voided in case:

- Our conditions in the operation manual are ignored
- The appliance is used beyond the described uses
- The appliance is modified or opened
- Mechanical damage or damage by media, liquids, natural wear and tear.
- Non correct implantation or assembly.

4.5 Monitoring of Test Resources

For body height measuring rods, we recommend a metrological examination of the accuracy of the measuring rod, however, this is not mandatory as the determination of human body height involves rather large, intrinsic inaccuracies.

4.6 Plausibility check

Please make sure that the measurement values computed by the appliance are plausible and are allocated to the respective patient, before storing and using the values for further purposes.

4.7 Reporting serious incidents

All serious incidents appeared related to this product must be reported to the manufacturer and the responsible authority of the member state where the user and/or the patient are residents.

„Serious incident“ that means an incident which directly or indirectly had, could have or could have had one of the following consequences:

- the death of a patient, a user or another person,
- the temporary or permanent fatal deterioration of the health status of a patient, a user or other persons,
- a serious danger for public health.

5 Basic Safety Precautions

	⇒ Carefully read this operating instructions before setup and commissioning, even if you are already familiar with KERN balances.	
---	---	---

5.1 Personnel training

The medical staff must apply and follow the operating instructions for proper use and care of the product.

5.2 Preventing contamination

In order to avoid crossed contamination (fungal disease,...) the parts coming into contact with the patient must be cleared regularly.

Recommendation: After every application which could have a potential contamination as consequence (e.g. direct contact with the skin).

5.3 Preparation for use

- Check the body height measuring rod for damage before any use
- Do not use the appliance on slippery surfaces or in facilities with risk of vibration
- If possible, the product must remain in its original packaging for transportation purpose. Should this not be possible, make sure that the product is protected against damage
- Only use the body height measuring rod when a qualified person is present.

5.4 Safety Instructions

- Observe operating instructions.
- Please use no other than the supplied fixtures for mounting.
- During assembling, ensure that screws are tightened properly and that the height measuring rod has been assembled correctly.
- For medical application recheck correctness of scale after assembly with the help of a calibrated length measuring device. (see chap. 6.4 „Checking the scale“)
- To prevent risk of injury, ensure that the head stop is returned to folded-in position after each use.
- The body height measuring emits only reliable values, when the back and the head are straightly aligned, nevertheless the body height of a person may change about some cm during a day.

6 Unpacking/installing

6.1 Testing upon acceptance

Check the packaging, as well as the body height measuring rod for external damage, right after having received.

6.2 Packaging / return transport

Keep all parts of the original packaging for a possibly required return. Only use original packaging for returning.

6.3 Scope of delivery

Scope of delivery	
	2 x bracket
	4 x dowel, "wall bracket"
	4 x screw, "bracket on stand scales"
	2 x screw, "measuring rod on stand scales"
	4 x screw, "Measuring rod on wall with bracket"
	Operating instructions

6.4 Checking the scale

After the assembly of the body height measuring rod, the scale must be checked using an external measuring rod.

- For this put an external, verified measuring rod with the respective accurateness on the weighing plate or, in case of wall assembly, vertically on the floor.
- The body height measuring rod must be shifted in a way that the horizontally unfolded flap touches the external measuring rod with the lower side at 150 cm.







Note: The measure of the external measuring rod does not coincide with the scale of the height measuring rod

- The scale is shifted by some centimeters because of the offset between flap and reading mark
- If the measure does not match exactly, the screws inserted during assembly should be once more slackened slightly and shifted accordingly
 - After this adaptation the scale of the body height measuring rod must be checked once more for the measure of 150 cm


6.5 Assembly

Wall mounting:


Set marking at 54 cm above the floor, using a verified length measuring device.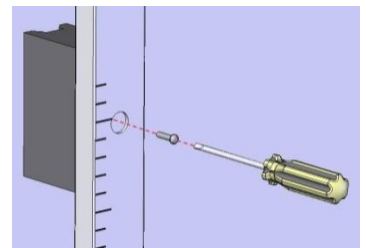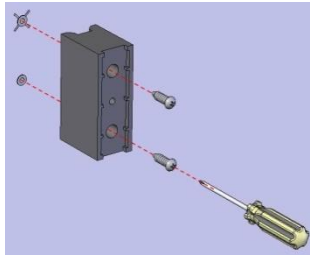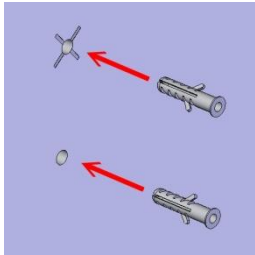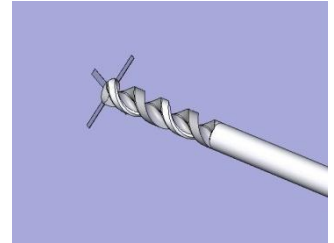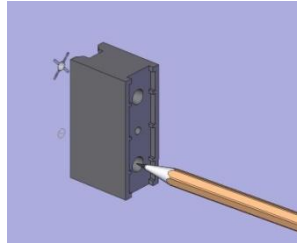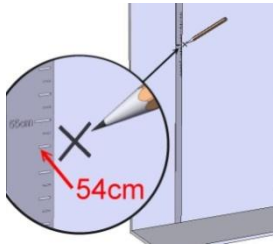

How to attach to KERN balances:

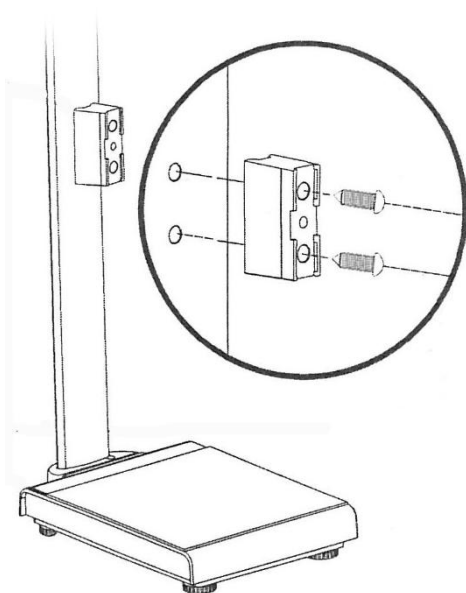Use the 2 screws for screwing the bracket into the supplied insert nuts on the stand of the balance.

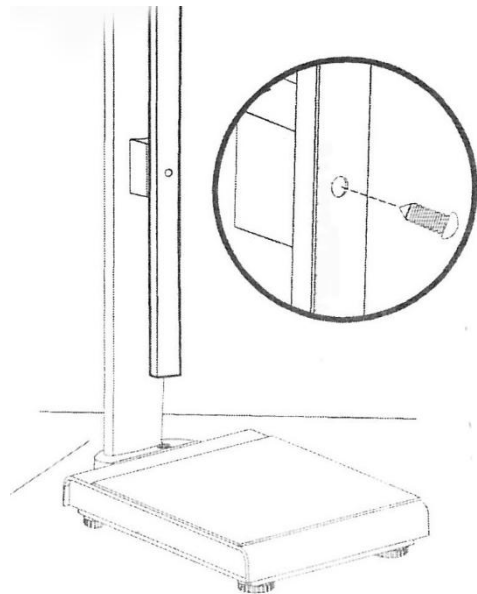


Extend the body height measuring rod and tighten it in the lower hole of the bracket with the help of the appertaining screw.

The body height measuring rod can be assembled on the backside of the stand in the same way.




7 Body height measurement

Body height above 120 cm: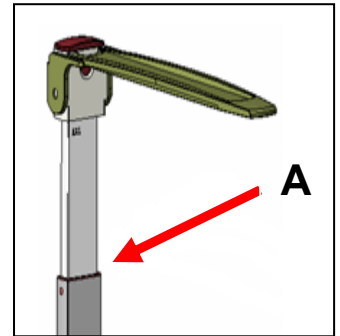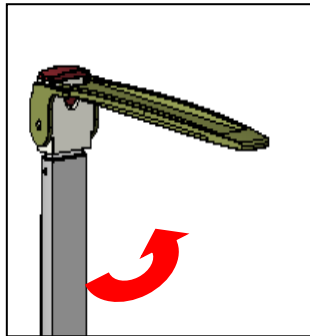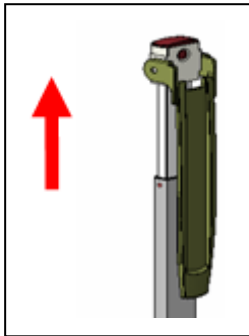

1. Pull measuring rod upwards
2. Move head stop up into horizontal position until it locks.
3. Carefully push the measuring rod down until the head stop gently touches the aligned head of the person. Read body height on the measuring rod (marking A).

Body height until 120 cm

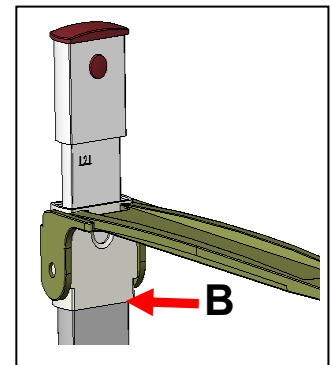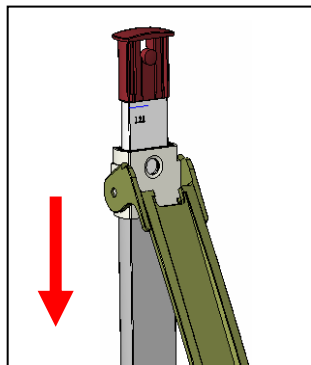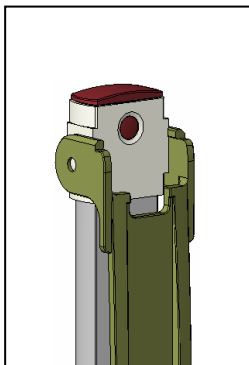4. Press locking and keep it pressed.
5. Shift head stop downwards.
Then move head stop up into horizontal position until it locks.
6. Carefully push the measuring rod down until the head stop gently touches the aligned head of the person.
Read body height on the measuring rod (marking B).

8 Cleaning, Maintenance, Disposal

8.1 Cleaning / disinfecting

Clean the body height measuring rod with household detergent or a commercially available disinfectant, e.g. 70% isopropanol.

We recommend a detergent suitable for wiping disinfection. Please follow manufacturer's instructions.

Do not use abrasive or aggressive cleaners such as spirits or alcohol or similar as they might damage the high-quality surface.

	⇒ Do not spray the body height measuring rod with disinfectant, just wipe it. ⇒ Remove dirt immediately
---	---

8.2 Sterilisation

Sterilization of the device not allowed.

8.3 Servicing, maintenance

The appliance may only be opened by trained service technicians who are authorized by KERN.

8.4 Disposal

Disposal of packaging and the body height measuring rod must be carried out by owner-operator according to valid national or regional law of the location where the appliance is used.

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