Alkoholtester 4 in 1

Art.Nr. 10204

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Alcohol Tester 4 in 1

Item-No. 10204

This product complies with the applicable National and European specifications.

Intended Use

The intended uses of the alcohol tester include measuring the alcohol level in exhaled breath. This does not allow any conclusions to be made about the alcohol level in the blood. In no cases may the device be used to evaluate driving ability or the ability to perform potentially hazardous activities.

Only power the device by batteries (for correct type see „Technical Data“). The product may only be used indoors; never outdoors. Do not expose the device to damp, e.g. in bathrooms etc. Any use, other than that described above, may lead to damage to this product and involves other risks, such as short circuit, fire and electric shock. The safety instructions must be observed at all times.

Safety Instructions

The warranty will be void in the event of damage caused by failure to observe these safety instructions. We do not assume liability for any resulting damage. We do not assume any liability for material and personal damage caused by improper use or non-compliance with the safety instructions. The warranty will be void in such cases.

In the operating manual, an exclamation mark inside a triangle indicates important instructions.


The “hand” symbol indicates special information and advice on operating the device.

- For safety and approval (CE) reasons, unauthorised conversion and/or modifications to the product are not permitted.
- Drunkenness gives rise to slow reaction time and adversely affects discernment and judgement. Therefore, it is a potential risk while driving a motor vehicle or performing other hazardous activities.
- The alcohol tester measures the alcohol level in the exhaled air. This does not allow any conclusions to be drawn about the alcohol level in the blood and thus, must never be used to evaluate driving ability or the ability to perform potentially hazardous activities. Exact information can only be obtained by a blood test.
- The degree of drunkenness, at which a person is still able to drive, depends on many factors and also varies from person to person. Furthermore, it depends on body weight, the last meal, the amount, time and duration of the alcohol consumption, the general physical condition, health and many other factors. Thus, you should not drive after drinking alcohol consumption nor should you perform any hazardous activity. The alcohol tester is only intended to give you a rough indication of the alcohol level in your blood. It must not be used to determine driving ability or the ability to perform potentially hazardous activities.
- To obtain reliable measurements, the instructions in this manual must be strictly followed. However, faulty measurements may also arise due to polluted air or suchlike.
- We shall not accept liability for accidents or damage due to drunkenness.
- Only use the alcohol tester when you are not distracted by important processes such as the traffic situation and environment.
- Do not expose the device to high temperatures, dripping or sprayed water, strong vibrations or heavy mechanical stress.
- Do not try to adjust, open, dismantle or repair the device. In the event of a malfunction, contact an expert.
- Keep the alcohol tester out of the reach of children.
- Do not leave the packaging material carelessly lying around since it may become a dangerous plaything for children.
- Please, also take note any additional safety information in these instructions.

Overview

- (1) Time display
- (2) Alcohol tester display
- (3) POWER button
- (4) Mouth piece
- (5) MODE button
- (6) SET button
- (7) ADJ button
- (8) Battery compartment
- (9) RESET button

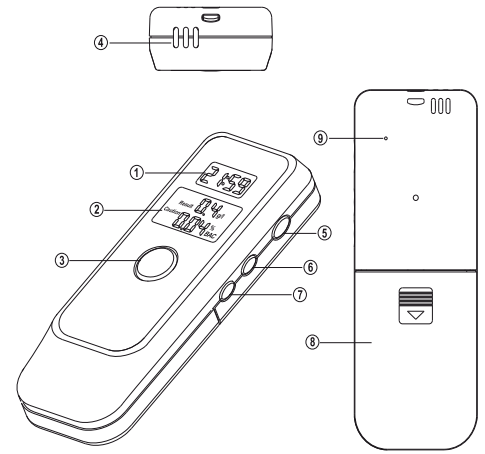
Inserting/Replacing the Batteries

Keep batteries out of the reach of children. Batteries are special waste and do not belong in the household waste. Ensure the polarity is correct, when inserting the batteries. Remove the batteries when the device is not in use for longer periods of time. Do not leave batteries lying around in the open; there is a risk of them being swallowed by children or domestic animals. If swallowed, consult a doctor immediately. The battery must not be short-circuited or thrown into a fire. Do not attempt to recharge the batteries. There is danger of explosion. Leaking or damaged batteries can cause burns if they come into contact with the skin. You should therefore wear suitable protective gloves.

- Open the battery compartment (8) on the rear of the device.
- Remove the insulation foil on the coin cell for the clock. This foil protects the coin cell from discharging during transport.
- Insert two new 1.5V micro-batteries (AAA) in the battery compartment, paying attention to the polarity markings. The correct polarity is given in the battery compartment.
- Close the battery compartment lid again.
- If the device does not work any longer, or when the symbol L appears in the display (2) of the alcohol tester, the batteries are flat and must be replaced as described above.
- When the coin cell for the clock is flat, replace it by a new one.
- Remove the batteries, if the device is not used for a while to avoid leaking or the batteries becoming flat.
- To ensure a long battery life, only use alkaline types.

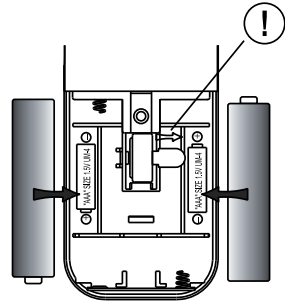
Operation

In order to get the most exact measurements, you should not eat, smoke or drink more alcohol 20 minutes prior to the reading. Also, avoid eating sweets or chewing gum. The measurement can be repeated to check the result. However, allow at least 3 minutes between the measurements. Keep the alcohol tester away from volatile substances such as paints, petrol, thinners, etc.

Warm up

- Press the POWER key (3) to switch on the device.
- WARM UP appears in the alcohol tester's display (2) and a countdown from 10 to 0.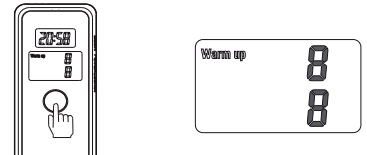

If you start up the device for the first time, or when it has not been in use for a while, we recommend the warm up process be repeated several times.


Measuring

- After the warm up process, the alcohol tester's display (2) shows BLOW.
- A countdown from 10 to 0 starts again.
- Blow into the mouthpiece (4) for approx. 3 to 5 seconds.
- After the measurement is completed, a signal will sound.
- The alcohol level is displayed.
- When the alcohol level is in the range of 0.2 to 0.5 g/l (per mill), CAUTION is displayed.
- When the alcohol level is in the range of 0.5 to 0.8 g/l (per mill) DANGER is displayed and a signal sounds.
- The value will be displayed for approx. 15 seconds, then the device switches off automatically.

Setting the clock

- Press the ADJ key (7) to switch between 12 h and 24 h-display.
- Press the SET key (6) for 2s, until the hours display blinks in the clock display.
- Use the ADJ (7) button to set the hour.
- Press the SET button (6) again, the minutes display flashes.
- Use the ADJ (7) button to set the minutes.
- Press the SET key (6) again to complete the setting.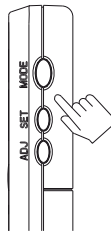

Alarm function

- Press the MODE key (5), until you enter the alarm mode (alarm clock symbol in the clock display (1)).
- Press the SET key (6) for 2s, until the hours display blinks in the clock display.
- Use the ADJ (7) button to set the hour.
- Press the SET button (6) again, the minutes display flashes.
- Use the ADJ (7) button to set the minutes.
- Press the SET key (6) again to complete the setting.
- Press the key ADJ (7) to switch the alarm function on or off. When the alarm function is activated, an alarm symbol appears between the hour and minute display.

Thermometer function

- Press the MODE button (5), to enter the thermometer mode.
- Press the ADJ key (7) to select °C or °F display.

Since the device warms up during operation, the temperature display, of course, can only display exact values if no alcohol readings have been taken just before measuring the temperature.


Further, holding the device in your hands may affect the displayed temperature.

Countdown Timer

- Press the MODE button (5), to enter the countdown mode.
- Press the SET key (6) for 2s, until the hours display blinks in the clock display.
- Use the ADJ (7) button to set the hour.
- Press the SET button (6) again, the minutes display flashes.
- Use the ADJ (7) button to set the minutes.
- Press the SET key (6) again to complete the setting.
- Press the ADJ key (7) to start or stop the countdown.
- When the preset time is up, a signal sounds.


Reset

- Press the RESET button (9) with a pointed object, to reset the clock to the factory settings.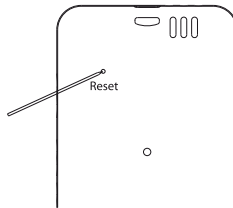

Attaching the wrist strap

- The wrist strap enclosed can be fastened to the alcohol tester by the thinner end, as shown in the picture.

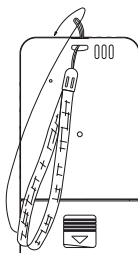Maintenance and Care

Apart from replacing the batteries, the alcohol tester is maintenance-free.

Only clean it with a slightly moistened cloth. Never use cleaning agents as these can damage the surface of the casing.

Ensure that no liquids enter the mouthpiece (4).

Do not store the alcohol tester in rooms with polluted air.

Disposal

Please dispose of the product, when it is no longer of use, according to the current statutory requirements.

Disposal of used batteries/rechargeable batteries

As the end user, you are required by law (Battery Ordinance) to return all used batteries/rechargeable batteries; disposal of them in the household waste is prohibited!

Contaminated batteries/rechargeable batteries are labelled with these symbols to indicate that disposal in the domestic waste is forbidden. The symbols for dangerous heavy metal constituents are: **Cd** = Cadmium, **Hg** = Mercury, **Pb** = Lead. You can return your used batteries/rechargeable batteries free of charge to the official collection points in your municipality, in our stores, or anywhere where batteries or rechargeable batteries are sold.

You thus fulfil your statutory obligations and contribute to the protection of the environment.

Technical Data

Operating voltage	3V= (alcohol tester) 1.5V= (clock)
Batteries for alcohol tester	2 x 1.5V micro battery (AAA)
Clock battery	1.5V coin cell AG13/LR44
Battery life	approx. 900 readings (alcohol tester) approx. 2 years (clock)
Measurement range	0.00-0.19% BAC 0.0-1.9 g/l (per mill)
Measurement accuracy	0.01% BAC 0.1g/l (per mill)
Warm up time	10s
Dimensions	100 x 37 x 17mm
Weight	50g (incl. Batteries)

Safety Instructions

Important: Damage caused by not adhering to the operating instructions is not covered by guarantee. HP-Autozubehör accepts no liability whatsoever for any resulting consequent damages.

- Only operate the device at its intended voltage.
- In case of repairs, only use original spare parts, to avoid serious damage.
- Check feed cables and conducting cables connected to the device for breaks or insulation defects. If you find a defect, or there is visible damage do not use the device.
- Repairs or other works, e.g. changing a fuse, etc. may only be carried out by an appropriately trained, skilled person.
- It must be noted that operator or connecting errors lie outside the influence of HP-Autozubehör and we cannot accept any liability for resulting damages.
- Always check whether the device is suitable for the respective place where you intend using it.

General information

1. Electric appliances, packaging material etc. should be kept away from children.
2. Dispose packaging and worn out material correctly and in an environmentally friendly way.
3. Do only pass on the device including the operation manual.
4. Subject to change of design and technical data without notice for the sake of constant product improvement.

Guarantee

We grant a two year guarantee for this device. Guarantee includes repair of all defects that were not caused by defective material or fabrication mistakes. Guarantee does not apply on wearing parts (for example illuminants). Since HP-Autozubehör has no influence on correct and appropriate assemblage and operation it is obvious that guarantee applies only on completeness and proper condition. HP-Autozubehör takes on neither liability nor guarantee for damages or consequential damages in connection with this product. This applies especially when the device was altered or repaired, when circuits were changed or non-original spare parts were used or when damage was caused by false or negligent operation or abuse.

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