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## Introduction

## GB

**Our sincere thanks** for deciding to purchase the digital TENS/EMS 4 channel electrical impulse stimulator.

The 4 channel electrical impulse stimulator is a new electrical stimulator to promote your daily well-being. The programmable impulses that are adapted to the physiological stimulation of the muscles and the nerves in the body are transmitted to the body by self-affixing pads placed in pairs on the body. Depending on the setting that has been chosen, the user can create a contraction (E.M.S= electrical muscle stimulation) or stimulation (T.E.N.S= transcutaneous electrical nerve stimulation) of the part of the muscle being treated. With the first twelve programs the user can vary the level of the intensity of 9 (+ and -). The stimulator also has four independent channels. Each channel requires two electrodes. The user can connect up to eight electrodes to stimulate for parts of the body simultaneously. Either batteries or mains supply can be used and so the stimulator can be used virtually anywhere without a break in treatment.

What exactly **can** the muscle stimulator do to help you?

- It can augment 'normal' muscle training
- By repeatedly maximising the muscular stimulation you will accelerate the recovery phase of that particular muscle and thus increase its strength.
- You will reduce the number of fat cells.
- You will shape your buttocks and thighs.
- You will relax the muscular tension, when you are over-taxing a muscle.
- You will obtain relief from nervous tension.
- You can rebuild your muscles that have become slack, e.g. after childbirth a woman can
  - strengthen her stomach muscles.
- make her facial muscles tighter.
- improve her blood circulation.

**The treatment will not produce the best results, unless you use the stimulator regularly. The results that you obtain can, however, differ from those obtained by another user.**

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#### Safety precautions

Before starting to use the stimulator for the first time you must read the instructions very carefully. Before you use the stimulator or switch it on you must, above all, take the following precautions: otherwise you will use the stimulator incorrectly:

- Carry out a visual check for **any signs of damage**;
- Ensure that the stimulator is **not near to anything containing a liquid**; the stimulator is not protected against the penetration of liquids (IP 20);
- Check that the **accessories needed for activating the stimulator** (electrodes, connections etc.) **are to hand** (please refer to the contents listed above.).

#### Side effects – contra-indications

The electrical impulse stimulator has been developed for use by an adult in good health. Special care needs to be taken under the following circumstances:

- Pace-maker (particularly a demand pace-maker)
- Serious heart rhythm abnormality
- Acute illness
- Epilepsy
- Psychoses
- Multiple sclerosis
- Inflammation of the veins
- Close to skin wounds and cuts
- Pregnancy
- Abnormal blood pressure
- High temperature

Nevertheless, the stimulator may be used in any of the above conditions, provided that your doctor agrees first. If you are in any doubt concerning the use of the stimulator or are suffering from any pain of unknown cause, you should consult your doctor.

When you are being medically treated for any reason, the stimulator should only be used under the instructions of your doctor.

The period allowed for recovery after you have used the stimulator should always be at least as long as the period you have used it. This precaution will ensure that the muscle will again be completely relaxed before the next massage.

Everyone needs a certain amount of time before the stimulator can use the stimulator on a higher setting. In particular, if your muscles have been subjected

to little or no exercise, you must keep the contraction time short and the recovery period longer.

#### WARNING

- When the appliance is switched on and the pads are attached to the body, in order to check the skin resistance the appliance in the intensity level "0" generates little impulses which can be perceptible on sensitive spots of the body. This is technically necessary and does not represent malfunction.
- Do not attach the pads at the front of your neck: there is the danger of laryngospasms at the side of the neck due to the sensitive nature of the glomus carotis, nor should you attach them to your eyes, head, or genitals.
- You must avoid affixing the pads on any part of the skin that is cracked, inflamed, infected or wounded. Neither should you affix the pads on any fracture, sprain, area where there is a metal implant nor on parts of the body where movement is restricted.
- You must not affix the pads if your body is still wet after a bath or a shower, or if it is sweating.
- Do not use the stimulator in the bath or under the shower.
- Metal objects, such as necklaces and belts, must not come into contact with the pads.
- Do not use the stimulator whilst asleep or whilst driving a motor vehicle.
- Keep the stimulator out of the reach of children.

If you suddenly feel unwell while using the stimulator or if it shows any indication of not operating correctly, stop using it immediately.

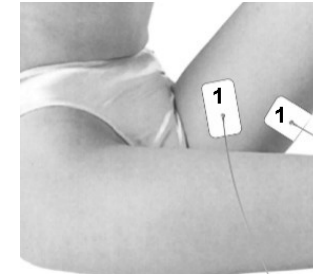
#### Warning

The stimulator is not intended for commercial or professional use in physiotherapy or dentistry.

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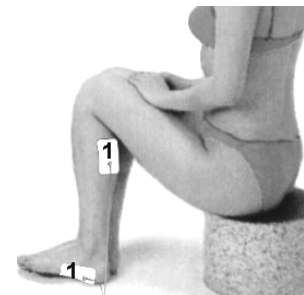
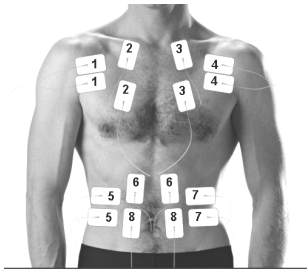
## Positioning the electrodes

(self-adhering pads)

- The stimulator must be switched off.
- Thoroughly clean the parts of the skin to which the electrodes are to be fixed and dry the skin thoroughly.
- Any hair that might be in the way should be removed before you use the stimulator, but do not shave the skin immediately before starting a treatment.
- Remove the protective foil from the pad and after cleaning the part of the body that is to be treated press the electrode firmly down.

## Removing and storing the electrodes

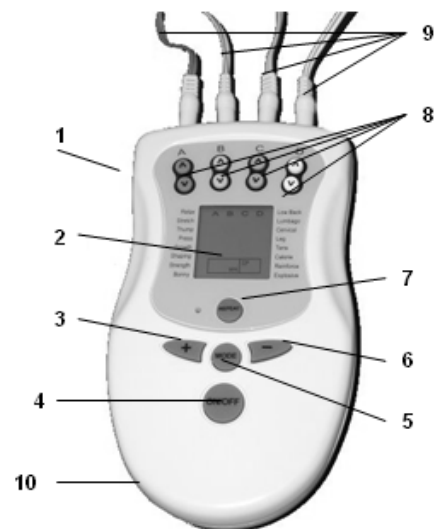
- Remove an electrode by pulling its edge; do not pull the ends of the leads, because this could damage the electrodes.
- When you are removing an electrode always pull in the same direction as the hair is growing.
- Replace the electrode on its protective foil again for safe storage.

**WARNING:** The life of the pads will vary: it will depend on skin type, condition of the user's skin, the level of stimulation that is applied or on environmental and storage conditions.

As a result of the skin contact of the pads the adhesion of the pads will gradually lessen as a result of the natural body fats that are exuded. You can ensure that the pads will grip over a longer period by wiping the pads with a damp cloth each time after switching off the stimulator. We suggest a change of pads after using them 30 times.

## The controls


- 1 - Socket for mains lead
- 2 - LCD display
- 3 / 6 - +/- (Program selection / Time (T) stimulation level (SP))
- 4 - ON / OFF button
- 5 - MODE (function selection: program, time, stimulation level)
- 7 - REPEAT button
- 8 - Setting: intensity for channels A / B / C / D
- 9 - Leads to channels A / B / C / D
- 10 - Battery compartment (on back)

## Instructions for use

1. Thoroughly clean the area you are going to treat.
2. Connect the stimulator to the mains using the connection provided (1) and to the plug socket or put the batteries in the battery compartment (10) on the back of the controls (as shown in the diagram inside the compartment).
3. Insert the leads into the appropriate sockets (9) using as many leads as are necessary (depending on the application: 1 to 4 leads).
4. Join the self-adhering pads with the clips on the leads you are going to use.
5. Remove the protective foil from the self-adhering pads.
6. Position the pads on the area that is to be stimulated: follow the examples showing positioning on Pages 9-11. (The poles for the plugs is indicated on the clip.). Press the electrodes down for a few seconds so that the gel adheres properly.
7. Press the ON-/OFF button (4) to start the stimulator. In Display (2) you will then see the initial setting: **Program 1 – 30 Min – 1**. Display (2) will light up. After 20 seconds this light will switch off automatically. You can turn it on again by pressing any button.

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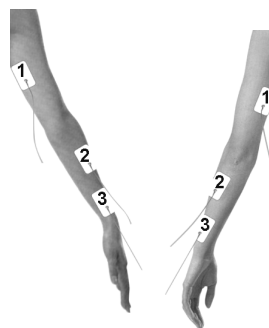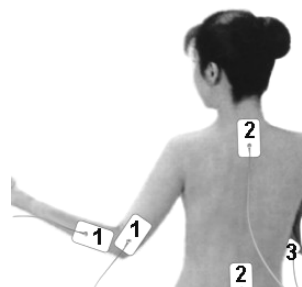
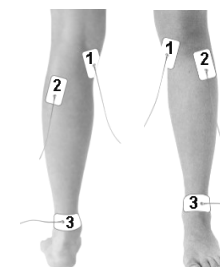
Taille:

aanbevolen programma's – 1, 2, 3, 9, 10, 12, 14, 21, 24, 26, 27, 28 und 30

8. **Selecting a program:** Press MODE (5) until the "Program number " appears in Display, then select the program you want by pressing +/- (3 + 6) (. Please refer to **Table 1**).
9. **Setting the time:** Press MODE (5) until you see "TIME" in the Display (2), then use +/- (3+6) to set the time you want (between 1 and 99 minutes).
10. **Setting the stimulation level (only P.1-12):** Press MODE (5) until you see "SPEED" in the Display (2), set the stimulation level you want (from 1 to 9) using +/- (3 + 6).
10. **Intensity (10 levels for each channel):** channels A, B, C, D. Press the UP/DOWN (8) buttons to set the channels. (You must set the intensity only of those channels to which leads are connected.)

**WARNING: If a pad for a channel does not make perfect contact with the skin, you will be unable to set the intensity for that channel.**

### IMPORTANT INFORMATION:

- a. After a few minutes you may feel that the intensity has intensified or weakened. If this happens, you can increase or decrease the intensity.
- b. When you are conditioning your muscles, you should increase the intensity in stages for the entire duration of the program (up to the maximum intensity that you can tolerate).

**WARNING: If you feel uncomfortable or are in pain while using the stimulator, you must reduce the intensity or stop treatment immediately by pressing the ON/OFF button (4).**

11. **REPEAT button (7):** When you are using programs 13-30, the stimulation level is variable (changing frequencies and intervals). Press the Repeat button (7) to lock the parameters applicable to the current setting. Press the Repeat button (7) a second time to continue the program.

12. While you are using the stimulator, you can select a different program (by repeating the procedure from Step 8) or switch off completely by pressing the ON/OFF button (4).

The stimulator has 30 pre-set basic programs: of these, programs 1-12 permit you to regulate impulse frequency by up to 9 increasing levels. Each program produces different effects and you can use each for a specific application. Table 1 lists the programs and the application for each of them.

**Table 1: Programs**

NO.	PROGRAM	APPLICATION	DURATION
P.1	Light stimulation	First stages of treatment with the stimulation; Muscle build-up after long period of recovery from injury	45-60 min
P.2	Aerobic stamina	Training for muscles used in aerobics and similar sports	10-15 min
P.3	Massage	Relaxation, recovery, warming up, capillarisation	45-60 min
P.4	Modelling	Lipolysis fat burning / cellulitis	45-60 min
P.5	Tensioning	Slack muscles, loose skin	30-45 min
P.6	Toning	Stomach, legs, arms	30-45 min
P.7	Blood circulation / lymph drainage	Improvement of and stimulation of local circulation	45-60 min
P.8	Endurance	Endurance of physical strength (middle distance, hills, etc.)	30-45 min
P.9	TENS chron. pain	Chronic pain, highest level for more depth effectiveness	45-60 min
P.10	TENS acute pain	Acute pain, highest level for more depth effectiveness	45-60 min
P.11	TENS varying	Varying frequencies (without patient's nerve system growing accustomed to stimulation)	45-60 min
P.12	Endorphinic TENS	Endorphinic effect: slower in taking effect but of longer duration	45-60 min
P.13	TENS muscles 1	Loosening effect / muscular pain	45-60 min
P.14	TENS muscles 2	Acute muscular pain / intense attacks of cramp	45-60 min
P.15	Fat burning	Localised reduction of fat	30-45 min
P.16	Training simulation	Fitness studio - strengthening	10-15 min
P.17	Training simulation	Endurance – physical strength	10-15 min
P.18	Lipolysis	Destruction and reduction of adipose cells	45-60 min
P.19	Definition	Muscle build-up and definition	10-15 min

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P.20	TENS relax 1	Relaxation during end-of-day fatigue	45-60 min
P.21	TENS relax 2	Evidence of fatigue / pain	45-60 min
P.22	Post-accident rehabilitation	Muscle stimulation to assist rehabilitation	45-60 min
P.23	Mixed stimulation	Build-up of muscle function	10-15 min
P.24	Insufficient mass	Fat burning	45-60 min
P.25	Muscle development	Mass and muscle definition	10-15 min
P.26	TENS mixed 1	Acute pain – long-lasting effectiveness	45-60 min
P.27	Strengthening	Build-up of muscle strength	30-45 min
P.28	TENS mixed 2	Chronic pain – long-lasting effectiveness	45-60 min
P.29	Maximum strength	Increase in maximum strength	10-15 min
P.30	TENS burst *	TENS + intervals for constant effectiveness	45-60 min

### Frequency of treatment

If the ailment is acute and the pain is particularly severe, it will often be of great benefit to repeat the pain therapy 2-3 times on the same day with an interval of a few hours. Then, in our opinion, the number of weekly treatment sessions should be as follows:

**Pain therapy** – daily until the pain ceases or becomes tolerable (if pain continues after 10 - 20 sessions a doctor should be consulted).

### Stimulation levels in programs 1-12

With programs 1-12 you can set the impulse frequency to 9 increasing levels (by selecting SPEED in the Display) and so have an influence of the effectiveness.

## Stimulation levels:

PROG	SP 1-3	SP 4-6	SP 7-9
P.1	Active recuperation	Prior to competition	Light tasks
P.2	Aerob. endurance - arms	Aerob. endurance - stomach	Aerob. endurance - legs
P.3	Relaxation	Massage/ warming up	Circulation/capillarisation
P.4	Modelling	Local fat reduction	Cellulitis
P.5	Arm tensioning	Stomach tightening	Leg tightening
P.6	Arm toning	Stomach toning	Leg toning
P.7	Local circulation	Lymph drainage	Diffuse bruising
P.8	Physical endurance - arms	Physical endurance - stomach	Physical endurance legs
P.9	Chronic tendon pain	Chronic joint pain	Chronic spinal
P.10	Acute tendon pain	Acute joint pain	Acute spinal pain
P.11	TENS burst* - tendons	TENS burst - joints	TENS burst - spine
P.12	Endorph.** - tendons	Endorph. - joints	Endorph. - spine

The device switches off automatically after 30 minutes running. This operation time can be changed from 1 to 99 minutes by pressing the function button "mode" together with the +- button. (see time setting)

### NOTES:

(\*) burst: variable TENS stimulation to reduce pain in order to avoid the effects of habit on the nerve system which would reduce the effects of pain reduction treatment.

(\*\*) Endorphine: pain reducing TENS stimulation using low frequency with a slower but more enduring effect.

It is important for you to place the electrodes correctly, because this is probably one of the most important factors contributing to successful TENS therapy. You must first try out the alternative positions for the electrodes so that you can find the ideal method for a particular treatment cycle. Spend a little time and effort on experimenting with the positioning of the electrodes. The likelihood of using the stimulator successfully will be increased quite considerably.

Each person reacts differently to electrical stimulation, and your own needs could best be served by changing the standard settings. Even if initial results are not positive, you should experiment further.

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In the following sections we have explained a number of different ways of positioning the electrodes. Please try each of them out or ask your doctor so advise you on the technique that is best for you.

### Directly sequential positioning

This is the way in which electrodes are most usually positioned. They are placed close to or around the localised pain in such a way that the electricity flows through or round the painful area.

This is done by placing one of the pads one side of the painful area. This will be successful when the pain is at an extremity and has been localised as lying deep inside the tissue. If the pad is positioned to the front or to the rear of the limb that is to be treated, the current will flow completely through the limb and thus through the painful area.

### Dermatomes, myotomes und sclerotomes

These are areas of the body which are enervated by a spinal nerve. The method used for positioning the electrodes includes stimulation via the enervated area. This is located by positioning one electrode on the painful trigger point and the other on the trigger point where the nerve root meets the spine.

### Motors, triggers and acupuncture trigger points

Though these trigger points are to be found with a high level of tissue conductivity at different areas and can also vary in the theoretical application, their use as a trigger point for attaching electrodes is exactly the same as the techniques described above. The simplest technique is to position a pad directly on the trigger point where the pain is felt and to close the electrical circuit by positioning the second pad on a master trigger point on the side that is painful. The explanation for arranging the separate master trigger points is unnecessary. Should there be any need, however, these should be determined with the help of a professional or a doctor.

### Multiple positioning

The TENS stimulator has four channels that operate independently from each other and so you can choose between two simultaneously operating strategies for positioning the pads. It is possible to use two different strategies at the same time for positioning the electrodes. One channel can be used to stimulate the painful area and the second to use the trigger point therapy. The programs are always active for all channels simultaneously.

### Placement of the pads to target specific muscle groups

The pads from one outlet on the appliance always form one unit. The placement of the pads can be carried out individually. However, 1 1, 2 2, 3 3 or 4 4 must be put together.

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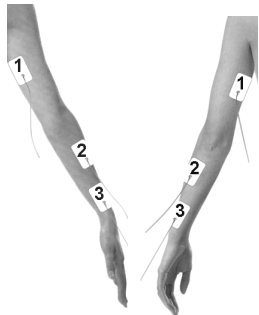
Feet:

Recommended programs – 2, 4, 6, 7, 22 und 30

## Cleaning and storing the stimulator and the accessories

- Please ensure that you have turned the stimulator off before starting to clean it.
- Clean down the outside of the stimulator with a mild household cleanser and wipe it with a soft cloth. Never immerse the stimulator in water and always ensure that it is never exposed to excessive dampness. Never allow any kind of liquid to enter the stimulator (e.g. via the lead connection): for this reason never use any kind of spray for cleaning it.
- Use a moist cloth to clean the lead. A light coating of talcum powder will prevent the lead from getting tangled and will lengthen its life.
- Magnetic fields and any form of electric radiation are to be avoided and you should store the stimulator away from a TV set and loudspeakers.
- Always store the device somewhere dry.

## Spare parts

1 set (8) self-adhesive pads Art.-Nr. 4513.1.30  
Connection lead Art.-Nr. 4513.1.32

### Contents:

- 1 control panel
- 8 self-adhering pads approx. 10x4.7cm with press-stud fastenings,
- 4 connection leads
- mains adapter 6V 600mA
- 4 batteries 1.5V AA Mignon
- instructions for use D / F / GB / NL / I

## Specifications

Model/type AD-2040  
Mains adapter: 230V AC / DC 6V 600mA  
Batteries : 4 x 1.5V AA Mignon  
Programs: 30  
Number of channels: 4  
Pulse frequency: 2 – 2000 Hz (adjustable)  
Waves: rectangular, two phase asymmetrical and  
TENS  
Dimensions: 170 x 96 x 36mm  
Weight: approx. 344g (without batteries) control  
panel only

## Guarantie

The guarantee lasts for 24 months from the date of purchase for manufacturers' error in material and quality. This guarantee is only valid if the instructions in the "instructions for use" have been followed. The guarantee is no longer valid if the device is damaged due to violent usage, misuse or improper use or service by unqualified persons.

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### Address for the guarantee and service:

**Hydas GmbH & Co. KG**, Am Hohlen Weg 37, D-34369 Hofgeismar, Germany

### Disposal of appliances and batteries

This product may not be disposed into household waste, but has to be entered the recycling of electrical and electronic appliances. As indicated with the symbol shown on the product, on the user manual or the packaging.

Please ask your local council for the nearest disposal facility.

**For the sake of the environment**, we request that you take packaging material and appliances that are no longer useable to be recycled (collection points).

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