ACE III **Basic**

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Operating manual

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Author liability:

All information herein was created conscientiously and under assumption of correctness. The creator cannot be held liable though for the use of any kind whatsoever and any ensuing consequences. The creator is not liable for any costs or expenses for damage that are the result of any information within this document. Any information is subject to change without prior notice.

Short description:

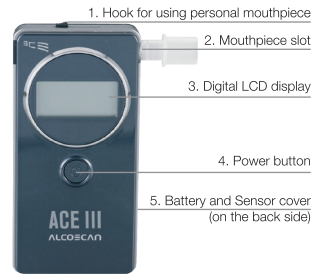
The ACE III Basic is a digital electrochemical alcohol testing device, which measures the alcohol concentration in the breath and calculates the blood alcohol level from that. The device can be operated by only one button. For each breath test a mouthpiece is needed. For a calibration service the device can be sent to ACE Technik.

Scope of delivery:

- Digital electrochemical breathalyser ACE III Basic
- Hard case
- 5 mouthpieces
- 2 pieces AAA batteries (1.5 V)
- Pouch
- Operating manual

Introduction:

Thank you for purchasing our breathalyser ACE III Basic. It measures alcohol concentration in human breath using an electrochemical sensor and offers reliable breath alcohol test results. It must only be used on basis of the instructions mentioned in this manual.

This way you receive a result as accurately as possible:

- a) Wait at least 15 minutes after your last sip of alcohol. Do not eat, drink or smoke during that time.
- b) Do not open the casing.
- c) Do not keep the ACE III Basic in cold, hot, moist or badly ventilated places.
- d) Keep the ACE III Basic out of the reach of children.
- e) Conduct three inlet tests after long storage before conducting the actual alcohol test.
- f) Do not let the ACE III Basic fall.

g) Change batteries after battery alarm. Diminished battery capacity leads to incorrect test results.

h) Do not conduct tests in loud (sound waves disrupt the sensor), draughty or badly ventilated places.

i) After a test result of 1.00 per mille or higher the sensor needs a resting time. In this case wait at least three minutes before the next test.

j) Do not conduct tests in very cold, very hot or unventilated, smoky ambience.

k) Store the ACE III Basic in a way to avoid penetration by dust or liquids.

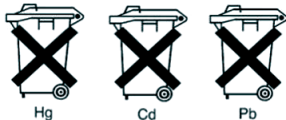
Notes to the battery law (BattG) :

The device for which this operating manual is contains two AAA alkaline batteries (1.5 V) in delivery condition.

Those batteries are not made for recharging. Trying to recharge can cause damage to property or persons. Also the use of rechargeable batteries is not possible with this device, because it can cause damage to the device. When the batteries are depleted, they must not be thrown in the household waste.

Used batteries may contain harmful substances that can cause damage to the environment or health. Please return batteries / rechargeable batteries to the store or to the buyback centre. The return is free of charge und mandatory by law. Please throw only depleted batteries into the containers and tape over the poles of lithium batteries. For battery exchange only pull off the battery compartment cover and change the depleted batteries for new ones. Mind the polarity: The negatives poles of the batteries are in contact with the springs in the battery trough. Slide the battery compartment cover over the casing until it snaps in. All batteries and rechargeable batteries will be recycled. This way, precious resources like iron, zinc and nickel are

gathered. Battery recycling is the easiest lesson in environmental.

The garbage bin means: Batteries and rechargeable batteries must not be thrown into the household waste. The signs below the garbage bins stand for:

- Hg: Battery contains mercury
- Cd: Battery contains cadmium
- Pb: Battery contains lead

Calibration:

To maintain the accuracy of the ACE III Basic, we recommend a calibration after 1 year at the latest. The calibration is not mandatory; the accuracy of ACE III Basic will decrease steadily without it, though. Send the device with the note "Alkomat calibration service" to the company address stated on the invoice you received with the device. We recommend sending it as a parcel, so that in cases of loss an inquiry or proof of delivery is possible.

Due to changes in the product, the procedures stated can change. This can take place without prior notice or adaptation of the operating manual and is no cause for complaint, return or price reduction. The user is responsible to recognise deviations between the operating manual and the actual handling of the ACE III Basic and he has to react accordingly.

Preparation for the test:

1. Install 2 pieces of AAA batteries (alka-

line, 1.5 V) in the battery compartment. Remove the battery and sensor compartment cover at the back of the device carefully and insert the 2 pieces of AAA batteries. Mind the battery polarity in the process (+/-).

2. For the active mode operation put a mouthpiece on the mouthpiece connection, which is on the side of the device. Make sure it is attached properly. Use a different mouthpiece for each test person. Avoid blowing cigarette smoke into the tester!

ACE Technik recommends:

■ Always use a new mouthpiece for each person and each test which is conducted. ACE Technik assumes no liability for any damages which arise when several people use the same mouthpiece for multiple tests.

■ Don't use the enclosed but the hygienic mouthpieces from the ACE series! During each test the ACE III Basic internal and external gets in contact with human secretion (saliva, blood, sweat etc.)!

■ Use safety gloves when pulling out a mouthpiece which was used by another person than yourself! You can buy appropriate gloves at ACE Technik.

Conducting the measurement:

1. Press the power button.
2. The device warms up until you can see the "blow" sign.
3. Blow through the mouthpiece into the unit for 4 seconds until the sounds (a "tictac" tone) concludes.
4. The result is displayed for 15 seconds. If you want to test again, press the power button after the back light has turned off.
5. The device automatically turns off after 4 minutes without using it.

If you keep press and hold the power switch, you can check the accumulated total test counts.

Important: When you have eaten, drunk alcohol or smoked a cigarette, wait at least 20 minutes before starting the measurement.

ACE Technik warns: Even if you get a result with the meaning „Safe range“, you should not drive after having consumed alcohol or do any other activities which are badly affected by the consumption of alcohol.

Usage of personal mouthpiece:

If you push the hook aside like shown on the right, you can use your personal mouthpiece.

Display & Meaning:

Flow: The breath sample was not strong enough for measurement. After warming automatically, please blow strong and steady during sampling.

bAt: The battery power is low. Please replace the batteries.

CAL: If you test over 1.000 tests, ACE III Basic will display "CAL" sign for 3 seconds when warming up for test. This "CAL" sign will be provided the appropriate time for calibration or replacement sensor module.

Err9: System error - Please let the device be repaired.

Technical data:

Statement of the BAC: 0,00 – 4,00 %
(fixed basic setting)

Accuracy: +/- 0,05 %

Warm-up time: 5 seconds up to 2
minutes

Response time: Within 10 seconds

Sensor: Fuel cell sensor

Energy supply: 2 pieces AAA alkaline
batteries (1.5 V)

Dimensions of the device: 50 mm x 105

mm x 15 mm (width x height x thickness)

Weight: ca. 85 g (inklusive Batterien)

Ambient conditions: Operation 5 – 40° C,
Storage 0 – 50° C

Calibration: Every 6-12 months or after
500 tests

Warranty and liability:

Warranty: The ACE Handels- und
Entwicklungs GmbH will give a warranty
of two years after purchase for material
and processing errors for the product,
to which this warranty and liability
declaration is attached, in all versions
and designations; a warranty of one year
for the sensor. This warranty includes
the electronics as well as the casing or
the sensor without sampling compo-
nents. Components that are inevitably
destroyed upon effectiveness (such as
fuses) are not part of this warranty. The
warranty is valid from the date of delivery
or from the date of first activation (always
the shorter period), if this is not stated
otherwise in the accompanying literature
of the product by ACE Handels- und
Entwicklungs GmbH.

Limitation of warranty and liability:

Product description: The ACE Handels-
und Entwicklungs GmbH rejects any
further expressly stated or implied
warranty, including the marketability and
suitability for a determined purpose. This

warranty especially excludes calibration.
After changing the sensor, calibration is
absolutely necessary.

When an exchange of the sensor or a
calibration is necessary for a product,
the user may receive a estimate of costs.
Should the user refuse the estimate, the
ACE Handels- und Entwicklungs GmbH
will send the product back unrepared /
uncalibrated and the product will be
labelled accordingly.

Should the product not fulfil the
warranty, the claim of the buyer and the
obligations of the ACE Handels- und
Entwicklungs GmbH are limited exclu-
sively and at the discretion of the ACE
Handels- und Entwicklungs GmbH to an
exchange or repair of the affected goods
or to a reimbursement of the purchasing
price for the faulty goods.

Application restriction: The ACE
Handels- und Entwicklungs GmbH is on
no account liable for additional damage,
incidental damage or consequential
damage such as, among others, lost pro-
fit and downtime that result from selling,
producing or using products, which are
part of this application restriction. This is
independent of any claims, such as from
contracts, negligence or other culpable
actions.

The product, which is accompanied
by this warranty and liability decla-
ration, no matter in which version or
designation, can especially not be used
legally binding to determine capability
to operate vehicles or to execute action
that could lead to damage to oneself or
others under the influence of alcohol.
The thorough check for damage of all
products upon receipt, a calibration
suitable for the given use by the buyer
and the proper use, maintenance and

service according to the instructions
of the accompanying product literature
by the ACE Handels- und Entwicklungs
GmbH are mandatory prerequisites for
this warranty. Maintenance and service
work by non-qualified persons, as well
as using non-approved consumables or
spare parts lead to an expiry of this war-
ranty. As for all technically challenging
products, it is a fundamental prerequisite
and mandatory for the warranty by ACE
Handels- und Entwicklungs GmbH, that
all users of these product are familiar
with the properties and restrictions of
the product as stated in the respective
literature.

Advice restriction:

The buyer confirms that he has checked
the intended use and suitability of
purchased goods for any purpose in
sole responsibility. Between the two
parties it is expressly agreed upon that
all technical or other advice by ACE
Handels- und Entwicklungs GmbH
regarding use of the goods or services is
free of charge and was made with to the
exclusive risk of the buyer. Therefore no
obligation or liability can arise to the ACE
Handels- und Entwicklungs GmbH from
any recommendation or their result.

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ACE Handels und Entwicklungs GmbH