## FT 78

<b>DE</b>	<b>Ohrthermometer</b> Gebrauchsanweisung .....	2-10
<b>GB</b>	<b>Ear thermometer</b> Instructions for use .....	11-17
<b>FR</b>	<b>Thermomètre auriculaire</b> Mode d'emploi .....	18-25
<b>ES</b>	<b>Termómetro para el oído</b> Instrucciones de uso .....	26-33
<b>IT</b>	<b>Termometro per orecchio</b> Istruzioni per l'uso .....	34-41
<b>TR</b>	<b>Kulak Termometresi</b> Kullanım Kılavuzu .....	42-49
<b>RUS</b>	<b>Термометр для измерения температуры в ухе</b> Инструкция по применению .....	50-58
<b>PL</b>	<b>Termometr do ucha</b> Instrukcja obsługi .....	59-67
	<b>ELECTROMAGNETIC COMPATIBILITY</b> <b>INFORMATION</b> .....	68-71

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

**Dear customer,**

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage and air. Please read these instructions for use carefully and observe the information they contain.

With kind regards,  
Your Beurer team

### **Included in delivery**

Check that the delivery has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 x ear thermometer
- 1 x button cell battery (CR2032)
- 6 x protective cap
- 1 x set of instructions for use

## Table of contents

1. Signs and symbols .....	12
2. Intended use .....	12
3. Notes .....	12
4. Unit description .....	14
5. Initial use .....	14
6. Settings .....	14
6.1 Switching the acoustic signals on and off .....	14
6.2 Switching temperature unit (Celsius °C -> Fahrenheit °F) .....	14
6.3 Memory function .....	14
7. Measuring .....	15
8. Replacing the battery .....	16
9. Cleaning and storage .....	16
10. Disposal .....	16
11. What if there are error messages? .....	16
12. Technical data .....	17

## 1. Signs and symbols

The following symbols appear in these instructions for use and on the device:

	<b>WARNING</b> Warning notice indicating a risk of injury or damage to health.
	<b>IMPORTANT</b> Safety note indicating possible damage to the unit/accessory.
	<b>Note</b> Note on important information.
	Observe the instructions for use
	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.
	Manufacturer
<b>IP 22</b>	IP 22, protection against harmful ingress of water and dust

## 2. Intended use

The ear thermometer is only intended for measuring the body temperature in the ear. Do not use the ear thermometer for measurements in other parts of the body.

The ear thermometer has been carefully developed for precise, safe and rapid temperature measurements in the ear. It is a non-invasive ear thermometer with an infra-red detector to determine the body temperature in the auditory canal of adults and children.

Use the device for the intended purpose only and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

The quality of the device has been verified and conforms to the requirements of Council Directive 93/42/EEC (Medical Devices Directive) Annex I, Essential requirements, and the corresponding harmonised standards. EN 12470-5: 2003 Clinical thermometers – Part 5: Performance of infra-red ear thermometers (with maximum device).

Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

## 3. Notes

### Safety notes

### WARNING

- To measure temperature, insert the sensor tip of the ear thermometer carefully into the ear.
- Use of the thermometer on different persons can be inappropriate in the event of certain acute infectious diseases because of the possible spread of germs despite cleaning and disinfection. If you have any doubts, please consult your doctor.

## Notes on handling batteries

### WARNING

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with plenty of water and seek medical assistance.
- Danger of being swallowed. Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Batteries must not be recharged, taken apart, thrown into an open fire or short circuited.
- Batteries can contain toxins that are harmful to health and the environment. Always dispose of batteries in accordance with applicable legal regulations. Do not dispose of batteries with the normal household waste.
- Observe the plus (+) and minus (-) polarity signs.
- Replace weak batteries before they discharge completely.
- Leaking batteries may damage the device. If you do not intend to use the device for longer periods, remove the batteries from the battery compartment.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

## General notes

### Note

- This ear thermometer is a sensitive electronic device. Handle it with care and do not subject it to any mechanical impacts.
- The ear thermometer is NOT waterproof. Keep it dry, and avoid contact with moisture, liquids, direct sunlight, high temperatures, high humidity and high dust concentrations.

- The ear thermometer has been designed for practical use, but is not a substitute for a visit to the doctor.
- The FT 78 ear thermometer is only designed for the measuring area on the human body stated in the instructions for use.
- The device is only intended for the purpose stated in these instructions for use.
- After each use, clean the measuring probe with a soft cloth moistened with disinfectant.
- Before each repair query, first check the batteries and replace them if necessary.
- Repairs must only be carried out by authorised service centres, otherwise the guarantee is invalidated.

### WARNING

- The body temperature measured in the ear may differ from other measurements taken in the mouth, rectum or armpits. Any comparison of these values is therefore impractical. Measure temperature regularly to calculate the normal ear temperature and then use these measurements as a basis for comparison when taking measurements in the event of a suspected high temperature.
- This thermometer is intended for measuring the temperature in the ear. Do not use this thermometer for measurements in other parts of the body.
- Replace the protective cover after each use to ensure an accurate measurement and prevent cross contamination.
- The thermometer may only be used with protective caps from Beurer; other protective caps may result in faulty measurements. If you have run out of protective caps, contact the manufacturer or retailer for replacements.
- Consult a doctor if you believe your medical condition to be poor.

## IMPORTANT

- The device is not shockproof. Do not allow it to be dropped and prevent any high impact stress on the device.
- Avoid bending the device.
- Do not take the device apart and do not modify it in any way.
- If this device is used according to the instructions for use, it is not necessary to recalibrate it regularly. If you still have any questions, please take the complete device to the retailer.

## 4. Unit description

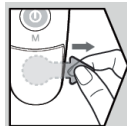
- 1 Protective cap
- 2 LCD display
- 3 ON/OFF/memory button (⏻)
- 4 Battery compartment
- 5 Start button (⏻)

- 1 Measurement symbol
- 2 Ear symbol
- 3 Battery symbol
- 4 Temperature unit (°C or °F)
- 5 Temperature display

If the measurement symbol (▶) and the ear symbol (👂) are visible on the LCD display, the device is ready to take a measurement.  
If (🔋) is shown on the LCD display, the batteries need to be replaced.

## 5. Initial use

The batteries are already inserted in all new devices. Before first use, remove the protruding plastic insulating strip from the battery compartment. The ear thermometer will then switch on automatically.

## 6. Settings

### 6.1 Switching the acoustic signals on and off

1. Ensure that the device is switched off.
2. Press and hold the start button (⏻) and then press the ON/OFF/memory button (⏻), until "C" appears on the LCD display.
3. To switch the acoustic signals on or off, press the ON/OFF/memory button (⏻).
4. Repeat these steps to switch the acoustic signals on or off.

### 6.2 Switching temperature unit (Celsius °C -> Fahrenheit °F)

1. Ensure that the device is switched off.
2. Press and hold the start button (⏻) and then press the ON/OFF/memory button (⏻), until "C" appears on the LCD display.
3. Press the start button (⏻) to switch from °C to °F.
4. You can switch the LCD display from "°F" to "°C" using the same process.

### 6.3 Memory function

There are a total of 9 memory groups for body temperature measurement. If the measurement values of the ear thermometer are in the range between 34 °C and 42.2 °C (93.2 °F–108.0 °F), the ear thermometer switches off and the latest measurement data is stored in the memory.

- Press the ON/OFF/Memory button (⏻) again to display the stored temperature.

## 7. Measuring

### Note

Check before each use that the lens is intact. If it is damaged, contact your retailer or the service address.

Bear in mind that the ear thermometer needs to have been in the room in which the measurement is taken for at least 30 minutes before use.

### WARNING

- Some people produce different readings in their left and right ear. In order to record temperature changes, always measure a person's temperature in the same ear.
- The ear thermometer may be used by children only under adult supervision. Measurement is usually possible over the age of 6 months. In infants under 6 months, the ear canal is still very narrow so the temperature of the eardrum often cannot be recorded and the result displayed is often too low.
- The measurement must not be taken in an ear affected by inflammatory diseases (e.g. discharging pus or secretion), after possible ear injuries (e.g. eardrum damage) or in the healing period after operative procedures. In all of these cases, please consult your doctor.
- Use of the thermometer on different persons can be inappropriate in the event of certain acute infectious diseases because of the possible spread of germs despite cleaning and disinfection. If you have any doubts, please consult your doctor.
- This ear thermometer may only be used with the protective cap in place.
- If you have been lying on one ear for some time, the temperature is slightly raised. Wait a little while or measure in the other ear.
- As ear wax can affect the measurement, you should clean the ear before measuring if necessary.

### Starting measurement

1. Twist a clean protective cap onto the device until you feel and hear it click firmly into place.

2. Press the ON/OFF/memory button to switch on the device (⏻).

3. As the ear canal is slightly curved, you have to pull the ear slightly up and backwards before inserting the sensor tip. This is important so that the tip of the protective cap can be pointed directly at the eardrum.

4. Insert the tip of the protective cap carefully and press the start button for 1 second (⏻). Release the start button (⏻). The end of the measuring time is signalled with a short beep and the measured value appears on the LCD display. If the measured temperature is above 37.5 °C (99.5 °F), the display flashes and four short acoustic signals are emitted. If the measured temperature is above 42.2 °C (108 °F), "Hi" appears on the LCD display and four short acoustic signals are emitted.

### Note

If you keep getting different measurement results, you should measure the temperature in the **test mode**.

To measure the temperature in the **test mode** proceed as follows:

- During the measurement hold down the start button (⏻) for three seconds.

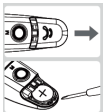
- Twist the measuring probe from side to side while taking the measurement to register the whole auditory canal or find the highest temperature. Release the start button (ⓘ) again when you hear a long acoustic signal. The measurement is complete.

5. For the next measurement, remove the old protective cap and put on a new one.
6. To switch the device off again, press and hold the ON/OFF/memory button (ⓘ) for approx. three seconds. If you do not press a button on the device for over one minute it will automatically switch off.
7. Clean the device after every use as described in the “Cleaning and storage” chapter.

## 8. Replacing the battery

Before replacing the battery, ensure that the device is switched off.

1. Open the battery compartment lid: Hold the device firmly and slide the battery out using a screwdriver.
2. Place the new battery under the metal flap and press the battery down until you hear it click into place.
3. Place the battery compartment lid back on.
4. When inserting the battery, ensure that the positive terminal (+) is facing upwards and the negative terminal (-) is facing downwards.

## 9. Cleaning and storage

The measuring probe is the most sensitive part of the ear thermometer. Work carefully when cleaning the measuring probe to avoid causing any damage. Use a clean cloth or cotton bud that can be moistened with disinfectant, alcohol or warm water.

To clean the entire device, use a soft cloth slightly moistened with a mild soapy solution.

Do not use any harsh cleaning products.

If prolonged storage is intended, you should remove the batteries.

The device must not be stored or used at an excessively high or low temperature or humidity (see technical data), in sunlight, in association with an electrical current or in dusty locations. Otherwise inaccuracies can occur.

## 10. Disposal

The empty, completely flat batteries should be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

Note: The codes below are printed on batteries containing harmful substances: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## 11. What if there are error messages?

Error message	Cause	Solution
	The ambient temperature is not between 10 °C and 40 °C (50 °F and 104 °F).	Keep the ear thermometer for at least 30 minutes in a room at between 10 °C and 40 °C (50 °F and 104 °F).

Error message	Cause	Solution
 bi-bi-bi	The device is not working properly.	Take out the battery, wait for one minute and then reinsert the battery. If the message is shown again, contact your retailer.
	The device cannot be switched on and will not switch to standby.	Insert a new battery.
 bi-bi-bi-bi	The measured temperature is over 42.2 °C (108.0 °F).	Check whether the protective cover for the measuring probe is damaged and take a new temperature measurement.
 bi-bi-bi-bi	The measured temperature is under 34 °C (93.2 °F).	Check whether the protective cover for the measuring probe is clean and take a new temperature measurement.
	Battery low; further measurements are not possible.	Replace the battery.
	Depleted battery: if the battery symbol is shown constantly, no more measurements are possible.	Replace the battery.

If you cannot find a solution to your problem here, contact our Customer Services.

## 12. Technical data

Temperature range	34.0–42.2 °C (93.2–108.0 °F)
Measurement accuracy	In the measurement range of 35.0 to 42.2 °C (95.0 to 108.0 °F): +/-0.2 °C (0.4 °F), in other indicating ranges: +/-0.3 °C (0.5 °F)
Operating temperature range	10–40 °C (50–104 °F), humidity ≤ 95%; 700–1060 hPa
Storage temperature range	-25–55 °C (-13–131 °F), humidity ≤ 95%; 700–1060 hPa
Memory	9 memory spaces
Display resolution	0.1
Battery	1 x 3V CR2032 battery
Weight	72 g including battery
Dimensions	153 x 42 x 35 mm
Protective cap	6 pcs
Automatic switch-off	60 seconds
Battery operating life	3,000 measurements in succession or one year with one to two measurements per day, including standby mode.
Safety classification	 Connected medical device of type BF
Clinical reproducibility	0.08 °C (< 1 year old), 0.10 °C (1–5 years old), 0.07 °C (> 5 years old)
IP classification	IP 22, protection against harmful ingress of water and dust

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# Electromagnetic Compatibility Information

**Table 1**

<i>Guidance and manufacturer's declaration-electromagnetic emissions</i>		
The FT78 Ear thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the FT78 Ear thermometer should assure that it is used in such an environment.		
<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment – guidance</b>
RF emissions CISPR 11	Group 1	The FT78 Ear thermometer uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The FT78 Ear thermometer is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

**Table 2***Guidance and manufacturer's declaration – electromagnetic immunity*

The FT78 Ear thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the FT78 Ear thermometer should assure that it is used in such an environment.

<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floor should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV air for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kV differential mode ±2kV common mode	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% $U_T$ (>95% dip in $U_T$ for 0.5 cycle 40% $U_T$ (60% dip in $U_T$ for 5 cycles 70% $U_T$ (30% dip in $U_T$ for 25 cycles 5% $U_T$ (>95% dip in $U_T$ for 5 sec	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the use of the FT78 Ear thermometer requires continued operation during power mains interruptions, it is recommended that the FT78 Ear thermometer be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
<b>NOTE:</b> $U_T$ is the a.c. mains voltage prior to application of the test level.			

**Table 3**

Guidance and manufacturer's declaration – electromagnetic immunity			
The FT78 Ear thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the FT78 Ear thermometer should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3Vrms 150 kHz to 80 MHz	3Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the FT78 Ear thermometer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance:</b></p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \text{ 80 MHz to 800 MHz}$
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	3V/m	$d = 2.3 \sqrt{P} \text{ 800 MHz to 2.5 GHz}$ <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p><b>NOTE 1</b> At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p><b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>Field strengths from fixed transmitters, such as base stations from radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast can not be predicted theoretic call with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the FT78 Ear thermometer is used exceeds the applicable RF compliance level above, the FT78 Ear thermometer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the FT78 Ear thermometer.</p> <p>Over the frequency range 150kHz to 80MHz, field strengths should be less than 3V/m.</p>			

**Table 4**

**Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM-for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING**

*Recommended separation distances between portable and mobile RF communications equipment and the FT78 Ear thermometer*

The FT78 Ear thermometer is intended for use in an electromagnetic environment in which radiated RF distances are controlled. The customer or the user of the T FT78 Ear thermometer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the FT78 Ear thermometer as recommended below according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.