Artikel | Item | Article  
Artikel | Articolo | Artículo

4508.1.00

4508GAz1\_1503

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## ENGLISH

GB

1	Introduction.....	20
1.1	Medical Background.....	20
1.2	What is TENS?.....	20
1.3	Important safety information.....	21
1.4	Intended use.....	21
1.5	Contraindications.....	21
1.6	Precautions.....	22
1.7	Side effects.....	23
2	General description.....	23
2.1	Description of the device.....	24
2.2	Key functions.....	24
2.3	Technical information.....	24
2.4	Program.....	25
3	Instruction for use.....	25
3.1	Possible areas for treatment.....	27
4	Cleaning and maintenance.....	29
5	Maintenance.....	30
6	Troubleshooting.....	30
7	Storage.....	31
8	Explanation of the symbols.....	31
9	Warranty.....	32

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# 1 | Introduction

The device 4508 is a small, portable device that consists of a power source, one signal generators and an adhesive electrode pad. It activates the body's own painkilling mechanisms, which sends low-intensity electrical pulses to the nerves below the skin relieving chronic pain in a range of cases.

It has been backed by studies conducted over the world that have proved that tens for pain is very effective and has none of the side effects.

The mini TENS 4508 is very easy to use and very discreet. Its small size makes it convenient to wear all day and this allows you to reduce pain and return to an active lifestyle.

## 1.1 | Medical background

### Explanation of pain:

Pain is a warning signal -we need these signals to tell us that something may be wrong with our body. Without this signal, we could not know that something in our body may be impaired or further damaged. However, once we have identified damage, pain serves little purpose.

Pain does not begin until coded message travels to the brain where it is decoded, analyzed, and then reacted to. The pain message travels from the injured area along the small nerves leading to the spinal cord. Here the message is switched to different nerves that travel up the spinal cord to the brain. The pain message is then interpreted, referred back and the pain is felt.

## 1.2 | What is TENS?

TENS (Transcutaneous Electrical Nerve stimulation) gives good results in acute and chronic pain conditions of many kinds. It is clinically proven and used daily by physiotherapists, other caregivers and top athletes around the world.

**High-frequency TENS** activates the pain-inhibiting mechanisms of the nervous system. Electrical impulses from electrodes, placed on the skin over or near the painful area, stimulate the nerves to block the pain signals to the brain. The pain is not perceived.

**Low-frequency TENS** stimulates the release of endorphins, the body's natural painkiller.

**TENS is clinically tested and approved with the following applications:**

- cervical vertebrae pain
- scapulohumeral peri-arthritis
- shoulder pain
- arm pain
- back pain
- lumbar pain
- dysmenorrhea
- hip joint pain
- muscular pain
- spasmodic pain
- sports injuries

## 1.3 | Important safety hints

Retain the manual for further use and forward it to other users!

### 1.4 | Intended use

This device belongs to the group of electrical stimulator system. It is a TENS device, which is intended for relieving or reducing acute and chronic pain.

### 1.5 | Contraindications

- This device should not be used when cancer cells are present in the treatment area
- Stimulator should not be applied over swollen, infected or inflamed areas or skin eruptions.
- Safety has not been established for the use of therapeutic electrical stimulator during pregnancy.
- Users with implanted electronic equipment, such as pacemakers and intracardiac defibrillators who have not got doctor's permission are not allowed to use the device.

**Do not use the device in following cases:**

- epilepsy
- serious arterial circulatory problems in the lower limbs
- abdominal or inguinal hernia
- if you have pains of unknown origin

### Caution!

The long-term effects of chronic electrical stimulator are unknown.

- Do not use the device if you are connected to, or in the vicinity of, high-frequency surgical or industrial equipment. This may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator. It may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Do not use the device in the direct vicinity of short-wave or microwave equipment, since this may affect the output power of the stimulator.
- In the process of stimulating and therapy, please do not allow metal parts of belts, wristwatches or necklaces touch the leaf electrode pads.
- Please don't use it in the bathroom or other place with high humidity.
- Please do not place pads near heart, genitals or diseased skin.
- Never use on the areas of the skin which lack normal sensation.
- This device is not intended to be used by persons (including children) with reduced physical, sensory or mental capabilities or lack of experience and knowledge, unless they are supervised by a person responsible for their safety or receive instructions on how to use the device.
- This TENS Device is designed for adults and should never be used on infants or young children. Consult your physician or other health care professionals before use on older children.
- Keep the stimulator out of reach of children.
- Consult your doctor if you have any doubt with the treatment.
- This device is for home use only.

## 1.6 | Precautions

- The stimulator is suitable for private use.
- Keep yourself informed of the contraindications.
  - Read, understand and practice the precautionary and operating instructions.
  - Observe the precautionary and operational decals placed on the unit.
- The instruction of use was listed; any improper use may be dangerous.
- Caution is advised for patients with suspected or diagnosed heart problems.
- Some patients may experience skin irritation or hypersensitivity due to the electrical stimulator. If rash develops or pain persists, please consult a doctor.

- Electrode pad placement and stimulator settings should be based on the guidance of prescribing practitioner.
- Effectiveness is highly dependent on proper pad placement and requires patience in its determination.
- Skin irritation in the area of the adhesive pad can occur with prolonged use.
- The electrode pad may only be placed on healthy skin.
- Avoid skin irritation by ensuring that good contact is achieved between electrode pad and skin.
- If the stimulator levels are uncomfortable or become painful, reduce the intensity to a comfortable level.
- Please don't use it when driving or sleeping.
- Never use the device in rooms where aerosols (sprays) are used or pure oxygen is being administered.
- Do not use it near any highly flammable substances, gases or explosives.
- Do not use sharp objects such as pencil point or ballpoint pen to operate the buttons on the control panel.
- Inspect connection of the pad before each use.
- Electrical stimulators should be used only with the electrodes pads recommended for use by the manufacturer.
- Please check that the electrode pad is clean and undamaged before using.

## 1.7 | Side effects

- Skin irritation or burning under the electrode pad may occur.
- Possible allergic skin reaction to tape or gel may occur.
- If the stimulator levels are uncomfortable or become painful, reduce the intensity to a comfortable level.

## 2 | General description

### Contents

- Stimulator
- Electrode pad
- User manual
- 2 x AAA alkaline batteries

## 2.1 | Description of the device (Fig.1)

- 1 2 Electrodes buttons
- 3 [ON/SET+] button
- 4 [OFF/SET-] button
- 5 Battery compartment
- 6 Electrode pad
- 7 Program name

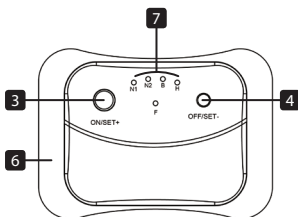
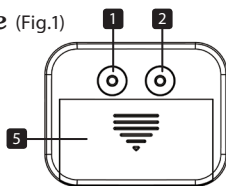
Fig. 1

## 2.2 | Key functions

**[ON/SET+] Button** = Turn on the device, device is in wait status (Diode glows permanent); in work status you increase intensity.

Press again the **[ON/SET+] Button** = the device is in work status in the selected Programm in level 1 (diode flashes);

Press again to increase the output intensity (to max. level 10).

**[OFF/SET-] Button** in wait status = Press the button to select the program; device is in work status = decrease intensity and turn off the device.

During operation, each press decreases the intensity by one level.

Press the button for 2 seconds to turn off the device.

## 2.3 | Technical information

Body material:	ABS
Channel:	Single channel
Power supply:	2x AAA alkaline batteries
Wave shape:	Biphase square-wave pulse
Pulse duration:	200 - 250 $\mu$ S
Pulse frequency:	1- 80 Hz
Output voltage:	max. 35 Vpp $\pm$ 20% at 500 Ohm load
Treatment time:	30 minutes
Intensity:	Adjustable from 0 to 10
Size of electrode pad:	53 x 73 mm

Operating conditions:	5°C to 40°C with a relative humidity of 30% - 85%, atmospheric pressure from 700 hpa to 1060 hPa
Storage conditions:	-10°C to 50°C with a relative humidity of 10% - 90%, atmospheric pressure from 700 hPa to 1060 hPa
Dimensions:	57 x 45 x 16,5 mm
Weight:	25 g (without batteries), 42 g (with batteries)
Classification:	B type applied part, internal power equipment

## 2.4 | Program

Program No.	Time (Min.)	Frequency (Hz)	Pulse width (uS)	Wave shape (TENS)
N1	30	80	200	normal
N2	30	2	250	normal
B	30	2	200	burst
H	30	2/80	250/200	acupuncture
F	30	1~80	200	modulation

## 3 | Instruction for use

- a** Open the battery compartment (Fig.2)
- b** Insert the batteries (2 x 1,5 V Type AAA).  
Pay attention to the correct polarity as marked inside the battery compartment (Fig. 2)

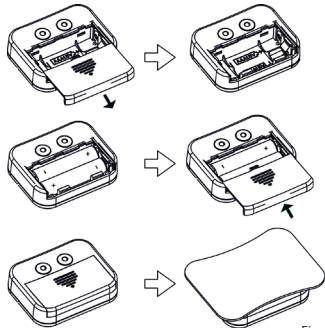
Fig. 2

### Note the safety instructions for batteries !

- Batteries may be fatal if swallowed. Therefore, keep the batteries and the product out of the range of children.
- If a battery was swallowed, consult a doctor immediately.
- Avoid skin and eye contact with leaking batteries. Wash affected areas immediately with plenty of clear water and consult a doctor.

- Batteries may not be charged, dismantled, thrown into fire or short-circuited.
- Protect batteries from excess heat.
- Remove the batteries when not using the unit for a long time, to prevent damage due to battery leakage.
- Always replace the full set of batteries. Do not mix old and new batteries or batteries of different types.

**c Connect the electrode pad to device.** (Fig.3)

Before proceeding to this step, be sure the device is completely turned OFF. Connect the electrode pad to the electrode button of stimulator correctly.

Fig. 3

**d Place the electrode pad on skin.**

Apply the electrode pad to the exact site indicated by your physician, following the instructions included with the electrodes labeling. Before applying the electrode pad, be sure the skin surface over which electrode is placed is thoroughly cleaned and dried. Make sure the electrode pad is placed firmly to the skin.

**Caution!**

- Do not switch the device on when the self – adhesive electrodes is not positioned on the body
- Never remove the self- adhesive electrode pad from the skin while the device is still turned on
- Only use the electrodes pad and lead wires that provided by the vendor. Using other products could lead to altered biological conditions and thus to false pulses.

**e Turn on the device**

Press the [ON/SET+] button to turn on the device.

**f Select the program**

Press [OFF/SET-] button to select the program (Fig.4); when the intensity has already been set, the program can not be changed.

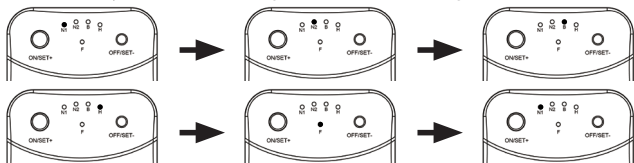
Fig. 4

**g Starting the treatment**

Press [ON/SET+] button to adjust the output intensity. The unit switches into »work status«. If the electrode pad is not placed firmly to the skin the device returns after 2 seconds into »wait status«.

**h Adjust the output intensity**

Press [ON/SET+] button to increase the output intensity (adjustable from 0 to 10). Press [OFF/SET-] button to decrease the intensity. The device will return to the »wait status« when the intensity is 0.

**i Turn off the device**

Press the [OFF/SET- ] button for 2 seconds to turn off the device at any mode.

**Batteries:** To replace the batteries, open the battery cover and remove the batteries. Replace them with two new AAA alkaline batteries.

## 3.1 | Possible areas for treatment


**Where to attach self-adhesive electrode pad?**



Each person reacts differently to electric nerve stimulation. Therefore the placement of the electrode differs individually. If application is not successful, contact your physician to find out which placement techniques are best for you. The size of adhesive pad may not be changed, e.g. by clipping off parts of them.

Note the following, when placing the electrodes for the treatment:

- If you want to activate the surface muscles, attach the adhesive electrode parallel to the fibres.
- To reach the deep muscle layers, place the adhesive pad diagonal to the muscle fibres.

Symptoms	Program	Possible areas for treatment
Cervical vertebrae pain	B, F	
Scapulohumeral periarthritis	H	

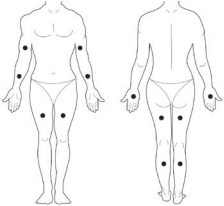Shoulder pain	B	
Arm pain	B	
Back pain	N1	
Lumbar pain	N1	
Dysmenorrhea	N1, H, B, N2, F	
Hip joint pain	N1, B	
Spasmodic pain	N2	

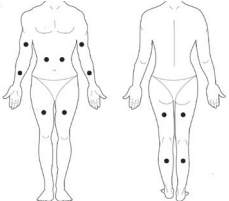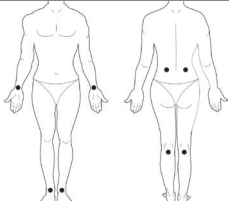Muscular pain	F	
Sports injuries	H, F	

## 4 | Cleaning and maintenance

The routine maintenance program below must absolutely be followed to ensure that the device operates adequately and under the conditions of maximum safety even long after the original purchase date:

### Cleaning the unit:

- Remove the batteries from the device every time you clean it.
- Use a soft, slightly damp cloth for cleaning.
- In case of more extreme soiling you can also moisten the cloth with mild cleaner.
- Do not immerse in liquids to clean the product. If water penetrates into the device, only use it again once it is completely dry.
- Do not use any chemical cleaners or abrasive agents for cleaning
- The user must not attempt any repairs to the device or any of its accessories
- If signs of damage are present or if the device was used improperly, stop using the product and contact the retailer.
- The device does not require calibration or verification of performance parameters.
- The characteristics are systematically verified and validated for each device manufactured.
- These characteristics are stable and do not vary when used under normal conditions.
- If the device does not work correctly, contact the Hydax customer service.

### Cleaning the electrode pad:

- Please use clear water to wash or use wet cloth (instead of facial tissue) to gently wipe up the electrodes when cleaning them.
- Do not use brush or fingernail to clean surface of the electrodes to avoid scratching them.

## 5 | Maintenance

- Manufacturers did not authorize to any maintenance agencies abroad. The manufacturer will not be held responsible for the results of maintenance or repairs by unauthorized persons.
- Do not drop, repair or remodel the unit.
- In case of repair please contact the retailer Hydas.
- Damage caused by excessive force, abuse, improper use or service by un authorized persons will void the warranty.

## 6 | Troubleshooting

The following table provides information on troubleshooting:

Malfunction	The reason	Countermeasure
LED does not light after replacing the batteries.	There is dirt on the battery compartment.	Check and clean battery compartment.
	Batteries are used up or inserted incorrectly.	Replace batteries or insert correctly
	There is dirt on the battery interface.	Check and clean battery interface.
	The wrong batteries have been inserted.	Replace batteries with AAA alkaline batteries.
There is no or only weak stimulation.	Do the electrode pads firmly stick to the skin?	Check connection and re-stick it if necessary.
	Is the connection between electrode pad and stimulator correct?	Check connection and re-connect if necessary.
	Batteries are used up.	Replace batteries.
	The electrode pad has been used too long and does no longer adheres well.	Replace electrode pad.
The treatment stops unexpectedly.	The electrode pad has loosen from body.	Check and re-stick the electrode pad.

The treatment stops unexpectedly.	Batteries are used up.	Replace batteries.
The skin in the treated area is red or tingling.	The treatment time is too long.	Do the treatment once a day. Shorten the treatment time if necessary.
	Electrode pad does not firmly stick to the skin.	Check if the pad adheres well to the skin.
	Interface of electrode pad is dirty or dry.	Replace the electrode pad if it is used up.
	The skin is sensitive to the adhesive of the pad.	In case of skin allergies or skin sensitivity change the point of treatment or reduce the time of treatment. In case of extreme skin sensitivity, the treatment should be stopped and a doctor should be consulted.

## 7 | Storage

- Remove the batteries from the device if you are not going to use it for a longer period. Leaking batteries can damage the unit.
- Do not make any sharp kinks in the connecting leads or electrodes.
- After use, stick the electrode pad onto the protective plastic film.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- Store the unit in a dry and clean place.
- Never place any heavy objects on the device.

## 8 | Explanation of symbols

Symbol for »Consult Operation Guide«

Symbol for »WARNING!«

Symbol for »TYPE BF applied parts«

Environment Protection - Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local Authority or retailer for recycling advice.

Symbol for »Manufacturer«

**SN** Symbol for »Serial number«

**CE 0483** Complies with MDD93/42/EEC requirements

## 9 | Warranty

The guarantee period is 24 months from the date of purchase and covers all manufacturers' errors in material and quality. This guarantee only applies if the instructions for use have been followed and becomes ineffective if force has been applied to the appliance or it has been used in any but the correct and proper manner, or if it has been serviced by any unauthorised person.

Dear Customer,

In case you product does not function as you expected please do not return immediately. Perhaps our hotline (+49-69-954061-24) can be of assistance. If you are still not satisfied, you have the possibility to return products free of charge within the BRD. We require your address. This you may provide to the above listed telephonic Hotline or by e-mail to: [technik@hydas.de](mailto:technik@hydas.de). Prepare the package for shipping and include the receipt and description of the problem. Please be aware of the fact that collect packages may not be accepted.

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# ELECTROMAGNETIC COMPATIBILITY INFORMATION

## For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacture's declaration - electromagnetic emissions		
The 4508 is intended for use in the electromagnetic environment specified below. The customer or the user of the 4508 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The 4508 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The 4508 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

## For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration - electromagnetic immunity			
The 4508 is intended for use in the electromagnetic environment specified below. The customer or the user of the 4508 should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	not applicable	not applicable (For INTERNALLY POWERED ME EQUIPMENT)

Surge IEC 61000-4-5	$\pm 1$ kV line(s) to line(s)	not applicable	not applicable (For INTERNALLY POWERED ME EQUIPMENT)
Voltage dips, short inter- ruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle  40% UT (60% dip in UT) for 5 cycles  70% UT (30% dip in UT) for 25 cycles  <5% UT (>95% dip in UT) for 5 sec	not applicable	not applicable (For INTERNALLY POWERED ME EQUIPMENT)
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels cha- racteristic of a typical location in a typical commercial or hospital environment.
<b>NOTE:</b> UT is the a.c. mains voltage prior to application of the test level.			

## For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

### Guidance and manufacturer's declaration - electromagnetic immunity

The 4508 is intended for use in the electromagnetic environment specified below. The customer or the user of the 4508 should assure that it is used in such an environment.

IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the 4508, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance:</p> $d = 1.2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3 \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup> Interference may occur in the vicinity of equipment. Marked with the following symbol:</p> 

**NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a</sup>: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast

and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the 4508 is used exceeds the applicable RF compliance level above, the 4508 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the 4508.

<sup>b</sup>: Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.

## For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

### Recommended separation distances between portable and mobile RF communications equipment and the 4508

The 4508 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the 4508 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the 4508 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0.12	0.12	0.23
0,1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

#### HERSTELLER

Hydas GmbH & Co.KG  
Hirzenhainer Straße 3,  
60435 Frankfurt/M., Germany  
Tel.:069/9540610,  
Fax: 069/95406140  
e-mail: info@hydass.de

#### GARANTIE & SERVICE

Hydas GmbH & Co. KG  
c/o atrikom fulfillment GmbH  
Haagweg 12  
65462 Ginsheim-Gustavsburg

#### OFFICE USA

Hydas Inc.  
P.O. Box 420, Hershey, PA. 17033  
Tel. 717-533-5583  
Fax 717-533-5548  
Warehouse: 1810 Church Rd.,  
Hummelstown, PA. 17036 U.S.A.  
e-mail: info@hydass.com