Produkt-Information „DISPOSABLE MEDICAL MASK“

Best.-Nr. 2251877

Best.-Nr. 2268666

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Hersteller: AAB(CHINA)CO.,LTD.

Adresse: AAB Industrial Park, Honglai Town, Nan'an City, 362000, Fujian Province, China

Tel.: +86-595-22980980

Email: sales@aab.com

http: www.aab.com

Name: Conrad Electronic SE

Adresse: Klaus-Conrad-Str. 1, D-92240 Hirschau, Germany

Tel.: +49(0)9622/30-0

Email: tkb@conrad.de

Product information „DISPOSABLE MEDICAL MASK“

Item No. 2251877

Item No. 2268666

How it works

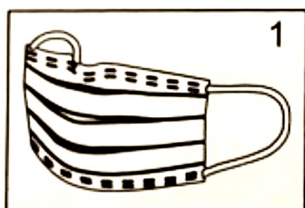
The product is made of 3-ply construction to help filtering $\geq 95\%$ of bacteria while remaining light and comfortable. The product is disposable and non-sterile.

Warnings

1. Avoid any touching the the inner side of the mask.
2. Check condition of mask, if visibly damaged, discard mask.
3. Mask is for single use only, read instruction before use.
4. Mask should not be used in a sterile surgical operation environment.

Method of use

1. Keep your hand dry and unfold the mask, hold it and keep the nose clip to the upside. Put loops around ears.
2. Adjust the face mask over nose and under chin, press the nose clip to make it well fixed to your nose.
3. Check for snug fit and comfort.

Model	A002
Product	DISPOSABLE MEDICAL MASK
Style	3 ply with ear loops
Size	175 mm x 95 mm
Color	Blue
Sterile	No
Materials	51% PP non woven + 25% melt blown non woven + 9% nose clip +15% ear loop
Latex free	Yes
Standard	YY/T0969-2013, EN14683:2019 Type I
Storage	Dry environment (<80% humidity), good ventilation conditions, away from direct sunlight
Manufacturing date (PRD DATE)	See marking on box
Expiration date (EXP DATE)	See marking on box
LOT number (LOT NO)	See marking on box

Manufacturer: AAB(CHINA)CO.,LTD.

Address: AAB Industrial Park, Honglai Town, Nan'an City, 362000, Fujian Province, China

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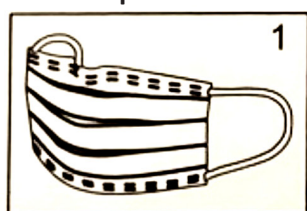
Produttore: AAB(CHINA)CO.,LTD.

Indirizzo: AAB Industrial Park, Honglai Town, Nan'an City, 362000, Fujian Province, China

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Fabricante: AAB(CHINA)CO.,LTD.