Produkt-Information „DISPOSABLE MEDICAL MASK“

Best.-Nr. 2252900

Best.-Nr. 2268532

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Hersteller: Guangdong Mideacom Medical Equipment Technology Co. Ltd.
Adresse: 10th Floor, Plant No. 14-15 of Fulonghsan, Chenquiao, Chaozhou City, Guangdong Province, China

Name: Conrad Electronic SE
Adresse: Klaus-Conrad-Str. 1, D-92240 Hirschau, Germany
Tel.: +49(0)9622/30-0
Email: tkb@conrad.de

Product information „DISPOSABLE MEDICAL MASK“

Item No. 2252900

Item No. 2268532

How it works

The product is made of 3-ply construction to help filtering $\geq 95\%$ of bacteria while remaining light and comfortable. The product is disposable and non-sterile.

Warnings

1. Avoid any touching the the inner side of the mask.
2. Check condition of mask, if visibly damaged, discard mask.
3. Mask is for single use only, read instruction before use.
4. Mask should not be used in a sterile surgical operation environment.

Method of use:

1. Keep your hand dry and unfold the mask, hold it and keep the nose clip to the upside. Put loops around ears.
2. Adjust the face mask over nose and under chin, press the nose clip to make it well fixed to your nose.
3. Check for snug fit and comfort.

Model	6110, 8110
Product	DISPOSABLE MEDICAL MASK
Style	3 ply with ear loops
Size	175 mm x 95 mm
Color	Blue
Sterile	No
Materials	51% PP non woven + 25% melt blown non woven + 9% nose clip +15% ear loop
Standard	EN14683:2005 Type I
Storage	Dry environment (<80% humidity), good ventilation conditions, away from direct sunlight
Manufacturing date (PRD DATE)	See marking on box
Expiration date (EXP DATE)	See marking on box
LOT number (LOT NO)	See marking on box

Manufacturer: Guangdong Mideacom Medical Equipment Technology Co. Ltd.
Address: 10th Floor, Plant No. 14-15 of Fulonghsan, Chenquiao, Chaozhou City,
Guangdong Province, China

Name: Conrad Electronic SE
Address: Klaus-Conrad-Str. 1, D-92240 Hirschau, Germany
Tel.: +49(0)9622/30-0
Email: tkb@conrad.de

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Produttore: Guangdong Mideacom Medical Equipment Technology Co. Ltd.
Indirizzo: 10th Floor, Plant No. 14-15 of Fulongshan, Chenquiao, Chaozhou City,
Guangdong Province, China

Nome: Conrad Electronic SE
Indirizzo: Klaus-Conrad-Str. 1, D-92240 Hirschau, Germany
Tel.: +49(0)9622/30-0
Email: tkb@conrad.de

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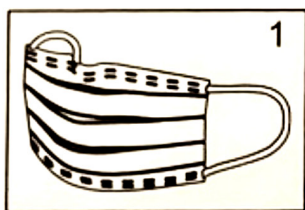
Fabricante: Guangdong Mideacom Medical Equipment Technology Co. Ltd.
Dirección: 10th Floor, Plant No. 14-15 of Fulonghsan, Chenquiao, Chaozhou City,
Guangdong Province, China

Nombre: Conrad Electronic SE
Dirección: Klaus-Conrad-Str. 1, D-92240 Hirschau, Germany
Tel.: +49(0)9622/30-0
Email: tkb@conrad.de