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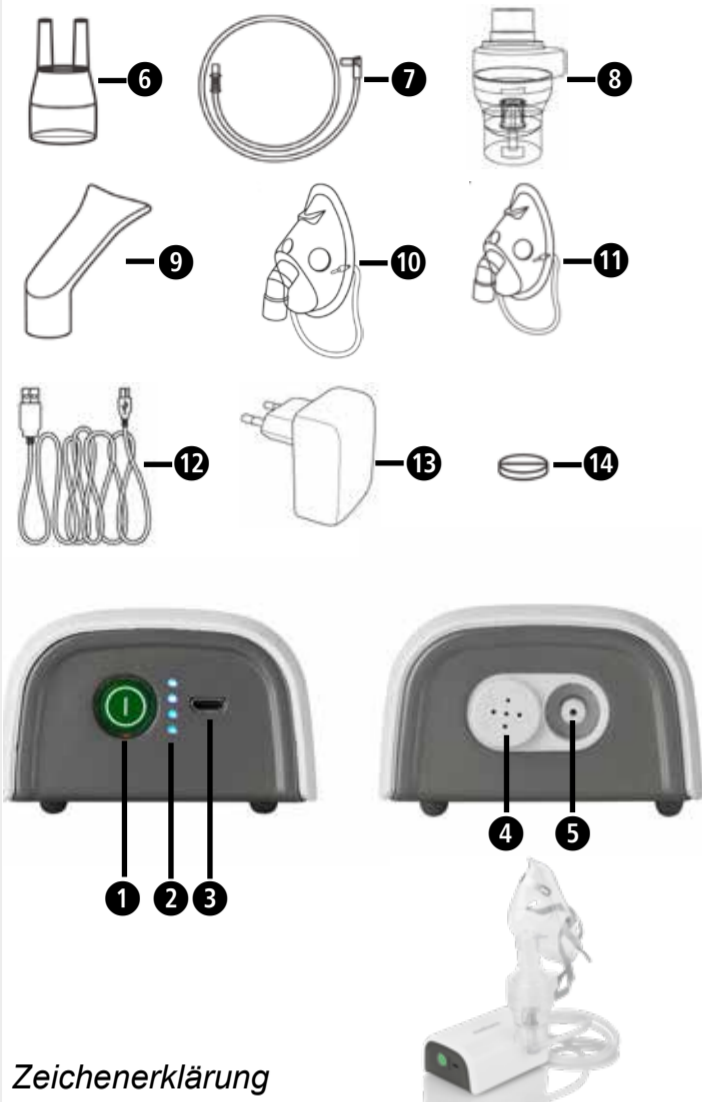
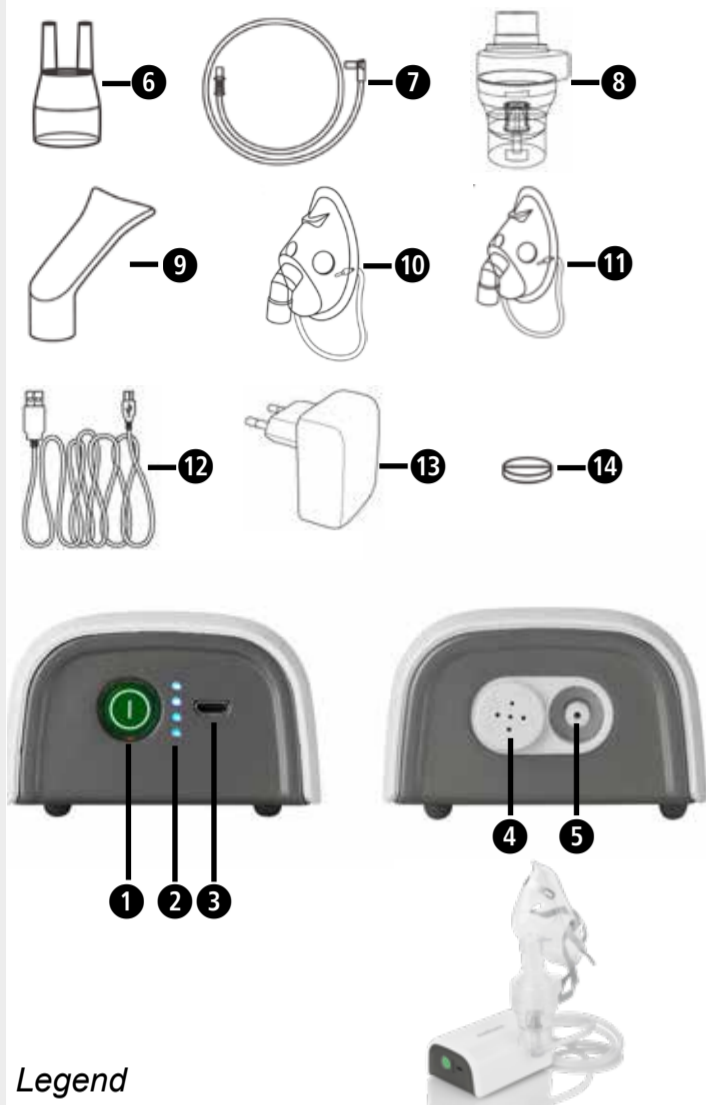
**EC REP**

Donawa, Lifescience Consulting Srl  
Piazza Albania, 10  
00153 Rome / Italy

## Device and controls

## Legend

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.

**WARNING**  
These warnings must be followed to prevent possible injury to the user.

**CAUTION**  
These instructions must be followed to prevent possible injury to the device.

**NOTE**  
These instructions provide you with useful additional information regarding installation or operation.

**IP21** Information about protection type against foreign objects and water

**Protection class II**

**LOT** Batch number

**Manufacturer**

**Date of manufacture**

O/I **Off/On**

**Authorised EU representative**

**Serial number of the device**

**0123**

## GB IMPORTANT INFORMATION! KEEP IN A SAFE PLACE!

**Read the instruction manual carefully before using the device, especially the safety instructions, and keep the instruction manual for further use. If you pass on the device to third parties, be sure to include this instruction manual.**

### Safety instructions

- This device must only be used for the purpose described in these instructions. The manufacturer is not liable for damage resulting from improper use.
- Do not use the device in the presence of flammable anaesthetic mixtures with oxygen or nitrous oxide (laughing gas).
- This device is only suitable for the anaesthesia and ventilation of the lungs.
- This device may only be used with the original accessories, which are listed in these instructions.
- Do not use the device, if you spot damage or you notice something unusual.
- Never open the device.
- This device consists of delicate components and must be handled with care. Observe the storage and operating conditions in the chapter "Technical Data".
- Protect the device from: - water and moisture, - extreme temperatures, - knocks and dropping, - dirt and dust, - strong sunlight, - heat and cold
- Adhere to the safety regulations applicable to electrical appliances, in particular the following:
  - Never touch the device with wet or moist hands.
  - Position the device on a level and stable surface during use.
  - Do not pull the power cable or the device to remove the plug from the socket.
  - The power cable plug is used to disconnect the device from the power supply, it must therefore always remain accessible during use.
- Before connecting the device, make sure that the electrical data on the label on the bottom of the device match the data of the mains.
- In case the mains plug of the device does not fit into the socket, contact qualified personnel to replace the mains plug. In general, the use of adapters and extension cables is not recommended. If it is essential to use them, then they must meet the safety regulations. In this case, however, the permitted limit values, which are specified on the adapters and extension cables, must always be adhered to.
- Do not leave the device plugged in when not in use; remove the plug from the socket when the device is not being used.
- The installation must be carried out in accordance with the manufacturer's specifications. Incorrect installation can cause damage to people, animals and objects, for which the manufacturer cannot be held liable.
- Do not replace the charging cable of this device. In the case of a faulty cable, contact a technical assistance centre approved by the manufacturer.
- The charging cable should always be fully unwound to avoid dangerous overheating.
- Before every cleaning or maintenance operation, the device must be switched off and the power cable removed from the socket.
- Only use the medicine prescribed to you by your doctor and follow the instructions of your doctor regarding dosage, duration and frequency of the therapy.
- Only use the parts specified by the doctor in accordance with your specific illness.
- Only use the nosepiece if expressly instructed to do so by your doctor. Make sure that the tubes are NEVER inserted into the nose, but are only held as near as possible in front of the nose.
- Check on the package insert of the medicine, whether there are contraindications for use with the usual systems for inhalation therapy.
- When positioning the device, make sure that the On/Off switch can be easily reached.
- For reasons of hygiene, do not use the same accessories for more than one person.
- Do not tilt the nebulizer by more than 60°.
- Do not use the device near strong electromagnetic fields such as mobile phones or radio equipment. Keep a minimum distance of 3.3 m to such devices when using this device.
- Make sure that children do not use the device unsupervised; some parts are so small that they could be swallowed. Tubes and lines of the device must be laid in such a way, that there is no risk of tripping, they are not kinked and the risk of strangulation is eliminated.
- The use of this device is not a replacement for visiting the doctor.

### Intended use

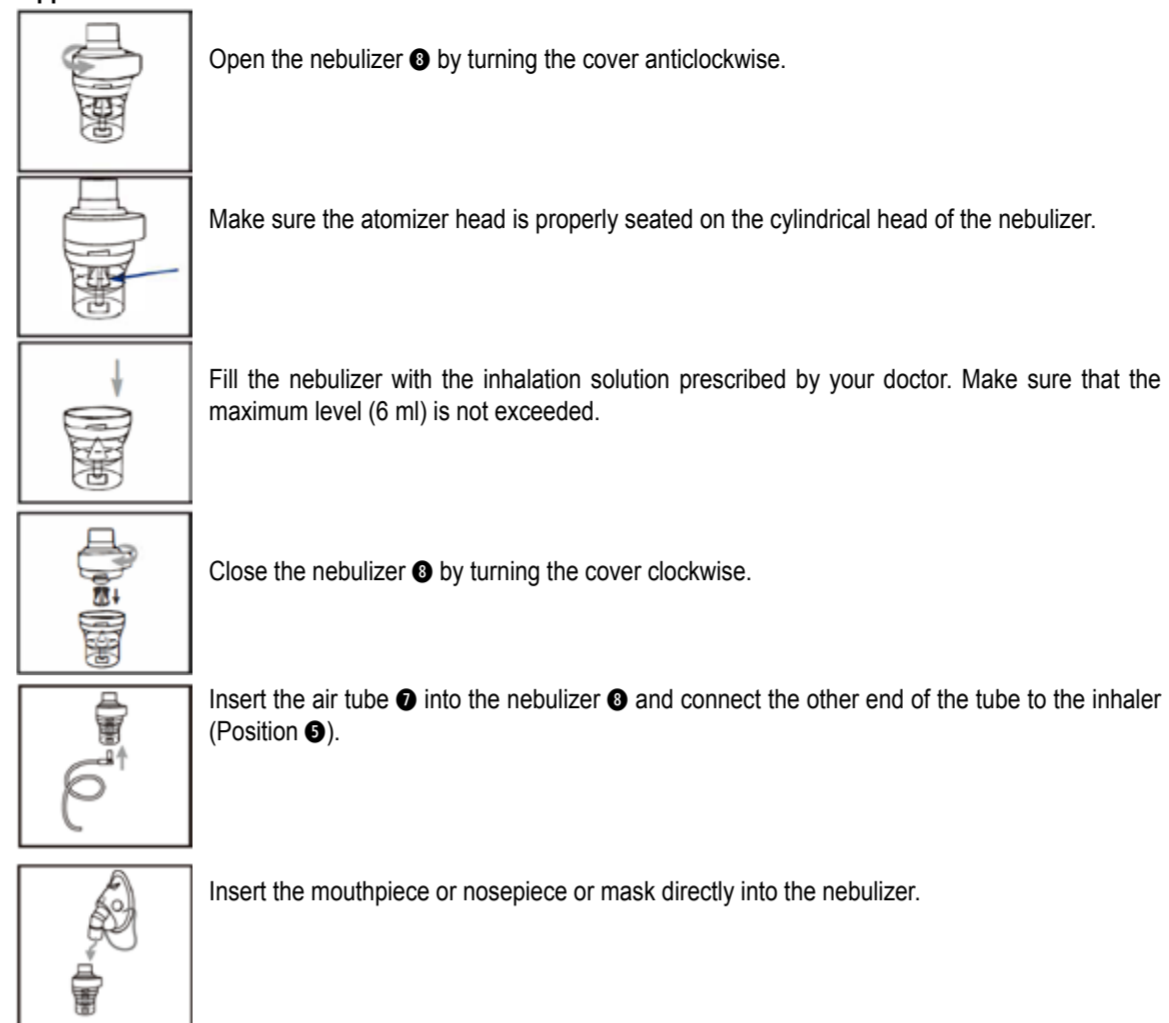
The MEDISANA inhaler IN 600/605 is an aerosol therapy system for home use. This device is designed for the nebulization of liquids and liquid medicines (aerosols) and for the treatment of the upper and lower airways.

### Preparing the device

Before the first use, we recommend cleaning all components - as described in the chapter "Cleaning and disinfection".

In between, you can charge the device with the USB cable ⑫ and USB charging adapter ⑬ included. To do this, connect the larger of the two connectors of the cable to the USB charging adapter and plug the other end of the cable into the charging socket ④ of the inhaler. The charge level is displayed with the 4 LEDs ①.

### Application

You have two options for the power supply of your inhaler.

Charge the device as described in 'Preparing the device' (for example for use while travelling) or use the cable and adapter as a direct power supply.

To start the treatment, briefly press the On/Off button ①.

While inhaling, sit upright and in a relaxed position at a table (not in an armchair), so as not to compress the airways and therefore not to impair the effectiveness of the treatment. Do not lie down during the inhalation. Stop the inhalation if you feel unwell. After you have finished inhaling for the amount of time recommended by your doctor, press the On/Off button ① in order to switch off the device and remove the plug from the electrical socket. Empty the remaining inhalation solution from the nebulizer and clean the device as described in the chapter "Cleaning and disinfection".

This device was developed for operation in 30 minutes On / 30 minutes Off mode. Please switch off the device after 30 minutes and wait a further 30 minutes, before you continue the treatment.

- The device does not require calibration. Modification of the device is not permitted.

**WARNING**  
Make sure that children do not get hold of the packaging films. There is a risk of suffocation!

### Scope of delivery

- 1 MEDISANA Inhaler IN 600/605 (① On/Off button, ② Charging indicator, ③ Micro-USB port (charging socket), ④ Position of the air filter, ⑤ Connection for the air tube)
- 1 Instruction manual
- Accessories: ⑥ Nosepiece, ⑦ Air tube (IN 605: two air tubes), ⑧ Nebulizer, ⑨ Mouthpiece, ⑩ Adult face mask, ⑪ Child face mask, IN 605: Baby face mask (without illustration)
- ⑫ Micro-USB cable (charging cable), ⑬ USB charging adapter, ⑭ 5 x Air filter (replacement at position ④), Storage bag (IN 605: storage pouch) without illustration.

### Cleaning and disinfection

- Clean all accessories thoroughly after each treatment in order to remove residues of medicinal product and possible impurities.
- Use a soft, dry cloth and a non-abrasive cleaning agent to clean the compressor.
- Make sure that there is no intrusion of liquids into the device and the power cable is disconnected.

### Cleaning and disinfection of the accessories

Follow the instructions for cleaning and disinfecting the accessories exactly, as they are fundamental for the performance of the device and the success of the therapy.

### Before and after each application

- Turn the upper part of the nebulizer ④ anticlockwise in order to open the nebulizer and remove the atomiser head.
- Wash all parts of the nebulizer, the mouthpiece ⑥ and the nosepiece ⑥. Then place in boiling water for 5 minutes.
- Wash the masks and the air tube with warm water.
- Assemble the nebulizer parts again and connect the nebulizer to the air tube.
- Switch the device on and leave in use for 10-15 minutes.

**Use only cold sterilisation solutions in accordance with the manufacturer's instructions.**

**Do not boil or autoclave masks and the air tube.**

### Maintenance and care

#### Replacement of the nebulizer

Replace the nebulizer ⑧ after a longer period of non-use, if it has deformations or cracks or if the atomiser head is blocked by dried-up medicine, dust, etc. We recommend replacing the nebulizer after 6 to 12 months depending on use. **Only use the original nebulizer!**

#### Replacement of the air filter

Under normal conditions of use, the air filter ④/⑤ should be replaced after about 100 hours of use or one year. We recommend regularly checking (10-12 applications) and replacing the air filter, if it is discoloured grey or brown or if it feels moist. Remove the air filter (Position ④) replace it with a new one. **Do not try to clean the filter for reuse. The air filter must not be repaired or maintained, while it is being used by a patient. Only use original filters! Do not use the device without a filter!**

### Malfunctions and countermeasures

*The device cannot be switched on*

- Make sure the charging cable is properly plugged in/Charge the battery.
- Make sure that the device has been operated within the operating period specified in these instructions (30 min. on / 30 min. off).

*The device is only misting a little or not at all*

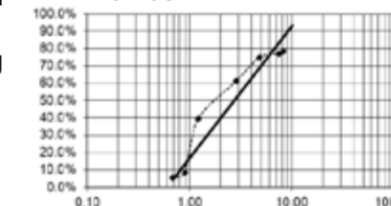
- Make sure that the air tube ⑦ is properly attached at both ends.
- Make sure that the air tube ⑦ is not compressed, bent, dirty or blocked. If necessary, replace it with a new one.
- Make sure that the nebulizer ⑧ is completely assembled and the atomiser head has been correctly positioned and is not blocked.
- Make sure that the required inhalation solution is filled in the correct amount (up to 6 ml).

### Technical data

Name	MEDISANA Inhaler IN 600/605
Power supply	Input: 100-240 V~ 50-60 Hz; Output: 5V DC, 2A
Nebulization amount (average)	0.25 ml/min.
Particle size	2.9 µm
max. pressure	1.1 bar
Noise level	45 dBA
Nebulizer fill quantity	min. 2 ml; max. 6 ml
Operating period	30 min. On / 30 min. Off
Expected service life	400 hours
Operating conditions	10 - 40 °C

### Storage and transport conditions

	10 - 95 % relative maximum humidity
	700 - 1060 hPa air pressure
	-20 - +60 °C
	10 - 95 % relative maximum humidity
	700 - 1060 hPa air pressure
Weight	240 g
Dimensions	10.8 x 7.2 x 4.4 cm
Length of the power cable	150 cm
IP class	IP 21
Reference to standards	EN 60601-1; EN 60601-2; 93/42/EWG
Item number	54542/54544
EAN number:	40 15588 54542 9/40 15588 54544 3

Aerosol properties according to EN 13544-1 APPENDIX CC  
Aerosol emission: 0.51 ml  
Aerosol emission rate: 0.16 ml/min.  
Particle size (MMAD): 2.9 µm

This device meets the requirements of the Directive concerning medical devices 93/42/EEC.

Device in Class II in relation to protection against electric shocks. Nebulizer, mouthpiece and masks are applied parts of type BF.

This device must not be disposed of with domestic waste. All users are obligated to bring all electrical or electronic devices to a collection point in their town/city or to a retailer, whether the devices contain any hazardous materials or not, so that they can be disposed of in an environmentally responsible manner. Please contact your local authority or retailer with regard to disposal procedures.

**In the course of constant product improvements, we reserve the right to make technical and design changes without prior notice.**

The current version of this instruction manual can be found at [www.medisana.com](http://www.medisana.com)

### Warranty and repair conditions

In case of warranty please contact your specialist shop or the service centre directly. If you need to return the device, please indicate the defect and attach a copy of the purchase receipt.

The following warranty conditions apply:

- All MEDISANA products are guaranteed for 3 years from the date of purchase. The date of purchase is to be proven in case of warranty by the purchase receipt or invoice.
- Defects due to material or manufacturing defects shall be repaired free of charge within the warranty period.
- A warranty service does not extend the warranty period for either the equipment or for the replaced components.
- The following are excluded from the warranty:
  - any damage caused by improper handling, e.g. by non-observance of the instruction manual.
  - Damage due to repair or intervention by the purchaser or unauthorised third parties.
  - Transport damage that has occurred on the way from the manufacturer to the consumer or has arisen on sending to the service point.
  - Accessories that are subject to normal wear and tear.
- Liability for direct or indirect consequential damages caused by the device is also excluded if the damage to the device is recognised as a warranty claim.

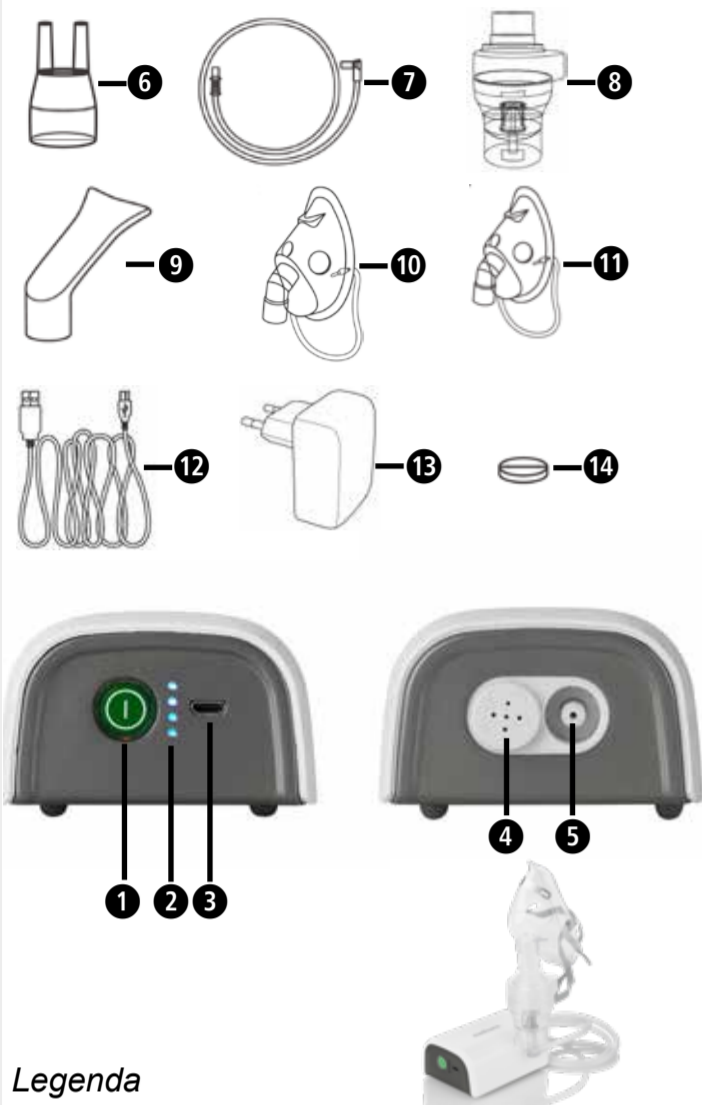
**Globalcare Medical Technology Co., Ltd**  
7th Building, 39 Middle Industrial Main Road,  
European Industrial Zone, Xiaolan Town  
528415 Zhongshan City, Guangdong Province  
PEOPLE'S REPUBLIC OF CHINA

imported & distributed by  
MEDISANA GmbH  
Jagenbergstrasse 19  
41468 NEUSS  
GERMANY

**Authorised EU representative**

Donawa, Lifescience Consulting Srl  
Piazza Albania, 10  
00153 Rome / Italy

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