INNOMARK gamma

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BREATHALYZER

Important Notes – EN

IMPORTANT NOTES ON THE CALIBRATION

Each breathalyzer which you buy from ACE Instruments can also be calibrated by ACE Instruments. Of course, we also calibrate devices which were bought from other contractors, as long as these devices are identical in construction to the devices sold by ACE Instruments.

You can see the due date of the next calibration on the calibration sticker placed on your breathalyzer. If the sticker is missing, the calibration is to be done at latest 12 months after the purchase date: e.g. purchase date: 13.05. = next calibration on the 13.11. and the next on the 13.05. of the following year.

For a calibration service you can send the device to the service address specified at www.ace-technik.com.

WHY IS A CALIBRATION NECESSARY?

The calibration is not mandatory, but the accuracy of the sensor installed in the breathalyzer will decrease steadily without a calibration and additionally guarantee expires, if you don't pay attention to the calibration ranges.

WHICH FACTORS INFLUENCE THE MEASUREMENT**ACCURACY**

- The more often the device is used the more the zero point is influenced.
- The higher the alcohol concentration is, the more the zero point is relocated.
- Devices which are used rarely have a decreased sensibility, too.

ACE Instruments recommends using the breathalyzer at least once (1x) a week independently of having drunken alcohol or not. To maintain the measurement accuracy, the breathalyzer should also be calibrated periodically.

PLEASE ALSO BE AWARE OF THE SEVERAL FACTORS OF INFLUENCE ON THE MEASURING RESULT

- Alcohol amount
- Period of the consumption of alcohol
- Gender
- Age
- Weight
- Height
- Drinking habits
- Health state
- Vitality of the liver
- Size of the lung
- Admission of food during the consumption of alcohol
- Influence of medicine

BREATHALYZER

Important Notes – EN

IMPORTANT NOTES ON TEST EXECUTION AND SELF-ASSESSMENT

After the last sip of alcohol you should wait at least 15 minutes, as there may still be residual alcohol in the oral cavity, which influences the measurement result. Also smoking should be avoided before a test, as cigarette smoke harms the sensor.

For each test you should use a new mouthpiece – not only for hygienic reasons, but also because an already used mouthpiece can influence the next measurement result. The mouthpiece could contain residual alcohol, which would affect the new test result when the same mouthpiece is used again (this for example could show an alcoholization of a person who hasn't drunken any alcohol).

If „FLO“ appears on the display, the given breath sample was insufficient or ended too soon. Blow into the moutpiece for about 4 seconds constantly (without a break) – preferably as if you were whistling without a sound.

If the same person performs 2 tests consecutively, then wait at least 3 minutes between the two tests, as a too short waiting time may cause different measurement results. The air exhaled from the lungs is, after a test, no longer enriched with the same alcohol content as in the previous test. Therefore, an immediate second test will indicate a lower test result.

All measured values determined with this breathalyzer are always indicative and not legally binding.

WHY THE ESTIMATED ALCOHOLIZATION AND THE MEASUREMENT RESULT IN MOST CASES CANNOT MATCH

An alcohol test is affected by a lot of different factors which also affect the test result. Beside the gender, age, body size and body weight, also the body fat percentage and the amount of body fluid have a big influence.

In addition, the general physical and mental state or the current daily constitution have to be taken into consideration. Furthermore, the meals eaten before a breath test (type of content and level of the stomach) and the intake of medication may influence the measurement.

Other influencing factors are, of course, the type of drink (percentage of alcohol content in the beverage and its carbon dioxide and sugar content), the drinking rate and the individual subject's duration of alcohol build-up or degradation in the body.

Some people quickly have a high alcohol content in the blood, with other people it takes longer until an alcoholization is detected. About 40-50% of the consumed alcohol is in the body within 15 minutes, after 30 minutes it is 65% and after 70 minutes the previously consumed alcohol is 100% absorbed in the body. However, it should also be noted that some people – partly due to genetic factors – quickly reduce the alcohol content in the blood, whereas others still have an increased alcohol content in the blood after a longer period of time.

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AUTHOR'S LIABILITY

All information herein was created conscientiously and under the assumption of correctness. Hence, the author cannot be held liable for the use of any kind whatsoever and/or any ensuing consequences. Furthermore, the author is not liable for any costs or expenses for damage that result from any information in this document. Any information is subject to change without prior notice.

INTRODUCTION

Thank you for purchasing our INNOMARK gamma Breathalyzer! It measures the alcohol concentration in human breath using an electrochemical sensor and calculates the blood alcohol level. The test results are very reliable and have a tolerance of $\pm 0.08\%$ (per mill = g/L) with reference to 0.50% at 25°C ambient temperature. The measurement results are displayed with 4 digits and are interpretable 1:1 (for example the display shows the value $0.36 = 0.36$ per mill).

The sensor inside the device can be calibrated. To maintain the measurement accuracy of this device and the sensor warranty, it should be calibrated regularly. For a calibration service you can send the device to the service address specified at www.ace-technik.com.

The device must only be used pursuant to the instructions mentioned in this manual. All measured values determined by this breathalyzer are always a guide only and are not legally binding. Breath tests should be carried out using clip-on mouthpieces (one new mouthpiece should be used per test for reasons of hygiene and to avoid falsified readings).

NOTE TO COUNTRY-SPECIFIC MEASUREMENT UNITS

Customers in the EU receive the breathalyzer with ‰ (per mill) as default setting, in the UK the default setting is mg/L, and in the USA, it is % BAC. In the chapter „Technical data,“ you will find information about the accuracy concerning the different measurement units. Please check the website of the Department of Transportation, the website of the Ministry of Transport, or automotive clubs to become informed about the current alcohol limits in your country.

SCOPE OF DELIVERY

- 1 Digital INNOMARK gamma Breathalyzer with Fuel Cell Sensor
- 1 rechargeable Lithium-Ion-Battery (3.7V)
- 1 Micro USB Cable (USB DC 5.0V)
- 3 Mouthpieces
- 1 Quick Start Guide (in German & English)
- 1 Manual (multilingual)

LABEL OF INDIVIDUAL COMPONENTS

1. Mouthpiece and Mouthpiece Slot
2. Digital LC Display
3. Power Button
4. Air Outlet (always keep this part unblocked)
5. Menu Buttons (to view the last 20 test results)
6. Connection for the USB Charger Cable
7. Button to switch on the power supply

TIPS TO RECEIVE THE MOST ACCURATE RESULTS

- A) Wait at least 15 minutes after your last sip of alcohol. Do not eat, drink, or smoke during that time.
- B) Do not open the casing.
- C) Do not keep the breathalyzer in cold, hot, moist, or badly ventilated places.
- D) Keep the breathalyzer out of the reach of children.
- E) Conduct three inlet tests after long storage before conducting the actual alcohol test.
- F) Do not drop the breathalyzer.
- G) Change the battery if it is depleted. Diminished battery capacity leads to incorrect test results.

- H) Do not conduct tests in loud (sound waves disrupt the sensor), drafty, or poorly ventilated places.
- I) After a test result of 1.00 per mill (per mill = EU setting) / 0.10% BAC (% BAC = US setting) / 0.50mg/L (mg/L = UK setting) or higher the sensor needs rest time. In this case wait at least three minutes before starting the next test.
- J) Do not conduct tests in very cold, very hot, or an unventilated, smoky ambience.
- K) Store the breathalyzer in a way to avoid penetration by dust or liquids.
- L) Blowing too hard may cause damage to the sensor as saliva flows into it.

PREPARATION FOR THE TEST

1. Recharge the lithium-ion battery (3.7V) built into the unit using the supplied charging cable. When starting the charging process, „CH“ is shown on the display and the battery indicator flashes at the same time, both disappear after a few seconds. As soon as the charging process is completed, you will see the word „FULL“ on the display. Then push the button below the USB cable connection upwards (a green background appears, the device can now be used).
2. Insert a mouthpiece into the slot on the left front side of the breathalyzer. Ensure whether it securely fits or not. It is highly recommended to use an extra mouthpiece for every test for sanitary reasons.

CONDUCTING THE MEASUREMENT

1. Briefly press the start button on the front of the alcohol tester. The device beeps twice and „ALCO“ and the battery indicator appear on the display.
2. Press the start button again once briefly, „SH“ (= Sensor Heating) appears briefly, then „SH99“ and a countdown starts (SH99 to SH00). During this first countdown, the sensor is heated up and „WAIT“ appears in the top left of the display. Then a second countdown starts, counting from 30 to 0. During this countdown, the device beeps every second and the display alternates between „READY“ and „BLOW“.
3. As soon as you start your breath test, the device stops beeping and the display shows „BLOW“ (also during the breath test). Blow evenly into the previously attached mouthpiece for 4 to 5 seconds. Breathe out slowly and without interruption (as if you were whistling without a sound).
4. As soon as you hear a click sound and a double beep, the test is finished. If the breath sample was interrupted or too weak, „FLO“ appears on the display. In this case, perform the test again and press the start button once.
5. After a successful breath test, the instrument evaluates the measurement result. The display briefly shows „Any“ and then immediately the result in per mille (= standard setting of the measuring unit in the EU). The measuring result can be seen for about 60 seconds, then „ALCO“ appears again on the display. For a new test, briefly press the start button once.

Note: The device switches off automatically after approx. 90 seconds. For another test, simply press the start button once briefly. You can also switch off the device manually by continuously pressing the start button for approx. 4 seconds.

STORAGE OF THE LAST 20 TEST RESULTS

If you push the button for switching on the power pack (= the button on the right side of the unit) upwards or from the red to the green position, the power pack is activated. If you then press the power button of the unit, you will briefly see the number of all tests ever performed with the unit.

The up and down buttons on the front of the unit are only used to view the tests performed so far (up to 20 test results are stored). If no test results have been saved yet, „- -“ is shown on the display.

You can also check the stored records after a successfully performed breath test and view the results. If the display shows „ALCO“ and you press the „up“ button for 2 seconds, you will get to the overview. For example, if 10 tests have been performed with the device, 10 test results can be displayed:

- 11 When ALCO is shown on the display, press and hold the „up“ button for 2 seconds and the test result MEM #0 is displayed.
12. if you press the upper button again, the display jumps to MEM #1, 2, 3, 4 etc. If you press the lower button, it jumps to MEM #9, 8, 7 etc.

To exit the overview of stored test results, briefly press the power button. This will take you back to the „ALCO“ display (this also happens automatically after a few seconds). Press the power button again to start a new test. The breath sample can be given during the countdown from 29 to 0 or while READY/BLOW is shown on the display.

Caution: All stored data records (MEM #0-19) are deleted when the power supply is interrupted (= when the slider on the right of the unit is pushed down or moved to the red position).

ACE INSTRUMENTS RECOMMENDS

- Always use a new mouthpiece for each person and test that is conducted. ACE Instruments assumes no liability for any damages that may arise when several people use the same mouthpiece for multiple tests.
- Only use the provided individually packaged and hygienic mouthpieces. During each test the breathalyzer's internal and external parts come into contact with human secretion (saliva, blood, sweat, etc.)
- Use safety gloves when pulling out a mouthpiece used by another person! You can buy appropriate gloves at ACE Technik (www.ace-technik.com).

IMPORTANT NOTICE

- WHEN YOU HAVE EATEN, DRUNK ALCOHOL OR, SMOKED A CIGARETTE, WAIT AT LEAST 15 MINUTES BEFORE STARTING THE MEASUREMENT. ANY TEST CONDUCTED PRIOR TO THE EXPIRATION OF THE RECOMMENDED WAITING PERIOD WILL RESULT IN INACCURATE RESULTS.
- IF THE TEST RESULT IS BELOW 0.10‰ (PER MILL = g/L) OR 0.01% BAC (US setting) OR 0.05mg/L (UK setting) THEN THE DEVICE SHOWS 0.00 (% OR % BAC OR mg/L – depending on the setting) ON THE DISPLAY. THIS HAPPENS BECAUSE AT A LOW RANGE OF ALCOHOL CONCENTRATION IT IS POSSIBLE TO MEASURE OTHER MATERIALS IN YOUR MOUTH.
- This breathalyzer is a high quality and sensitive measurement device. To maintain the accuracy of the sensor, it should regularly be used. If the device is not used for an extended period, the sensor can dry out. ACE Instruments recommends using the breathalyzer at least 2 times per month (even if you have not been drinking alcohol).

- Due to changes in the product, the procedures stated can change. This can take place without prior notice or adaptation of the operating manual and is no cause for complaint, return or price reduction. The user is responsible for recognizing discrepancies between the operating manual and the actual handling of the breathalyzer and must react accordingly.

ONLY USE ORIGINAL ACE INSTRUMENTS MOUTHPIECES

- Guarantees test result accuracy
- Guarantees hygiene of you and the device
- Guarantees full warranty of this device

ACE INSTRUMENTS ADVISES

even if you get a result meaning „safe range“, you should never drive after consuming alcohol or do any other activities that are severely impaired by alcohol consumption and might endanger your health and safety or the health and safety of others.

DISPLAY INDICATIONS

DISPLAY	MEANING	ACTION
	The rechargeable battery is depleted.	Please charge the battery with the supplied USB cable. Note: Battery performance will last longer if you slide down the power activation switch (on the right side of the unit) when the unit is not in use.
<i>Flo</i>	The sensor did not have enough breathing air for an analysis.	Press the start button and start a new test by constantly blowing into the mouthpiece for 5 seconds without pausing (blow as if you were whistling without sound).
	The total number of breath tests performed has reached 500. The device is locked and needs to be calibrated. Alternatively, you can purchase a replacement sensor and use it to replace the locked sensor. The deinstalled locked sensor can be calibrated and then be used for further 500 tests.	To stop the blinking of the word „LOCK“ and return the device to normal operation, the user must contact the seller of the alcohol tester and have the device calibrated. You can send the instrument to the service address mentioned at www.ace-technik.com for calibration.
<i>Err</i> _{non}	The sensor module is defective or the sensor must be replaced.	Please contact the seller of your breathalyzer or the ACE Instruments service department (see www.ace-technik.com).

CALIBRATION

To maintain the accuracy of the breathalyzer, ACE Instruments recommends a calibration after 500 tests at the latest or after 12 months (depending which incident happens first). For a calibration service you can send the device to ACE Instruments (see www.ace-technik.com). The calibration is not mandatory, but the accuracy of the breathalyzer will decrease steadily without it and additionally the warranty for the sensor expires.

We recommend sending it as a parcel, so that in case of loss an inquiry or proof of delivery is possible.

BATTERY NOTICES

The device referred to in this manual is delivered with a rechargeable 3.7V lithium-ion battery. This battery can be recharged up to 200 times. The battery will last longer if you slide down the power switch (on the right side of the unit) when the unit is not in use. If the battery is defective or no longer rechargeable, please send the device to the service address given at www.ace-technik.com. Do not open the device on your own authority and do not attempt to replace the battery yourself. ACE Instruments advises that this will void the warranty.

TECHNICAL DATA*

Sensor	Fuel Cell Sensor
Accuracy	+/- 0.08‰ (per mill) with reference to 0.50‰ at 25°C (per mill = default setting) OR +/- 0.008% BAC (Blood Alcohol Concentration) with reference to 0.050% BAC at 25°C OR +/- 0.04mg/L with reference to 0.25mg/L at 25°C
Detection Range	0.000 up to 4.000‰ (per mill) OR 0.000 to 0.400% BAC (USA) OR 0.000 to 2.000mg/L (UK)
Warm-up Time	Within 15 up to 30 seconds at 0.5‰, it may vary depending on the value of per mill / BAC which was measured
Dimension	110 x 53 x 19mm (L x W x H)
Weight	82g (battery included)
Power Supply	1 piece 3.7V Li-ion battery (rechargeable, fully loaded with energy for up to 80 tests)
Working Temperature	0°C up to 55°C
Storage Temperature	0°C up to 55°C
Calibration	Every 12 months or after 500 tests at the latest. For a calibration service you can send the device to ACE Instruments (see www.ace-technik.com).

* The technical data are subject to change without prior notice for functional improvements.

WARRANTY AND LIABILITY

ACE Handels- und Entwicklungs GmbH (ACE Instruments) provides a warranty of two years after purchase for material and processing errors of the product (see attached warranty and liability declaration) in all versions and designations; and a warranty of one year for the sensor. This warranty includes the electronics as well as the casing and sensor without sampling components. Components that inevitably degrade over time (such as fuses) are not part of this warranty. The warranty is valid from the date of delivery or from the date of first activation (always the shorter period), if not stated otherwise in the accompanying ACE Instruments product literature.

LIMITATION OF WARRANTY AND LIABILITY

1. PRODUCT DESCRIPTION

ACE Instruments rejects any further expressly stated or implied warranty, including the marketability and suitability for a determined purpose. This warranty especially excludes calibration. After changing the sensor, calibration is absolutely necessary. If an exchange of the sensor or a calibration is required for a product, the user may receive a cost estimate. Should the user reject the estimate, ACE Instruments will send the product back unrepared/not calibrated and the product will be labelled accordingly. Should the product not fulfil the warranty, the claim of the buyer and the obligations of ACE Instruments are limited exclusively to the discretion of ACE Instruments for exchange, repair of the affected goods or reimbursement of the purchasing price for the faulty merchandise.

2. APPLICATION RESTRICTION

ACE Instruments is by no means liable for additional, incidental or consequential damage such as, among others, lost profit and downtime that result from selling, producing or using products that are part of this application restriction. This is independent of any claims emanating from, for instance, contracts, negligence or other culpable actions. The product, which is accompanied by this warranty and liability declaration (no matter the version or designation), in particular cannot be used in a legally binding manner to determine one's capability to operate vehicles or execute action that could lead to personal injury or injury to others under the influence of alcohol.

The thorough check for damage of all products upon receipt, calibration suitable for the given use by the buyer and the proper use, maintenance and service according to the instructions of the accompanying ACE Instruments product literature are mandatory prerequisites of this warranty. Maintenance and service work performed by non-qualified persons as well as the use of non-approved consumables or spare parts results in the voiding of this warranty. As for all technically challenging products, it is a fundamental prerequisite and mandatory for ACE Instruments warranty that all users of these product are familiar with the properties and restrictions of the devices as stated in the respective literature.

3. ADVICE RESTRICTION

The buyer is solely responsible and confirms that they have checked the intended use and suitability of purchased goods for any purpose. Between the two parties it is expressly agreed that all technical or other advice given by ACE Instruments regarding the use of goods or services is free of charge and made under risk to the buyer exclusively. Therefore, no obligation or liability can result to ACE Instruments from any recommendation or their consequences.

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ACE Handels- & Entwicklungs GmbH