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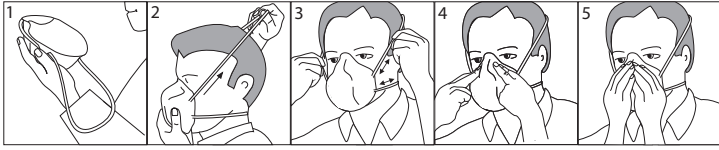
Feinstaubmaske FFP1 20 Stück (Best.-Nr. 2575494)

Feinstaubmaske FFP2 20 Stück (Best.-Nr. 2575493)

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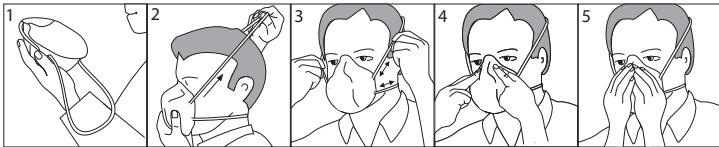
FFP2 Particulate Respirator 20 pcs (Item No. 2575493)

Compliance: EN149:2001+ A1:2009

These single-use particle filter half masks comply with the statutory national and European requirements. All company names and product names are trademarks of their respective owners. All rights reserved.

Warning

- Contact with skin may cause allergic reactions in people with sensitive skin. In this event, leave area of risk, remove mask, and seek medical advice.
- These operating instructions cannot replace a risk assessment or purchase advice. The user has to be trained and instructed in wearing the mask.
- When choosing and using the correct respiratory protection, a detailed analysis is required of the workplace and the hazardous substances used there.
- These respirators may not be used if the concentration type, and properties of contaminants in the ambient atmosphere are unknown or at dangerous levels.
- Respirators should be disposed if damaged, if the breathing resistance becomes high due to clogging or at the end of a single shift (up to 8 hours).
- Only remove the mask from packaging just before you are going to use it.
- The mask must be chosen according to the type of contaminant, its concentration and the protection factors as required by national laws.
- This mask must only be used once (no maintenance).
- Do not modify the mask and check for damage before using.
- Use the mask in well ventilated areas, where there is no lack of oxygen.
- Immediately leave the area if you feel dizzy, faint, or experience breathing difficulties.
- This mask DOES NOT PROTECT the user against gases or vapors and solvents from spray painting.
- Failure to observe these instructions may have adverse effects on the health of the user and expose them to serious and potentially life threatening risks.
- Do not use with facial hair, jewelry or eyewear that may inhibit contact between the face and the product thus preventing a good seal or application of the face mask.

Protection Class

- **FP1:** Filtering efficiency (80%), total inward leak (<22%), (FPA) Allocated Protection Factor of 4. Applications e.g. Handling stone, rubber, cellulose.
- **FP2:** Filtering efficiency (94%), total inward leak (<8%), (FPA) Allocated Protection Factor of 10. Applications e.g. Sanding soft wood, composite materials, rust, putty, plaster, plastics, cutting, deburring, grinding, drilling of metal.

Instructions for Use

1. Hold the mask in your hand, with nose bar facing towards the top and straps hanging down.
2. Cup mask under chin, and covering mouth and nose. Take upper headband and pass around the top of head, take lower headband and pass around back of neck.
3. Make sure straps are properly fitted.
4. Using two hands mold the nose bar around the nose and seal it off completely.
5. Check mask fit to ensure a good seal.
 - Gently cup both hands around the mask without disturbing mask fit, then INHALE sharply without pushing on the mask.
 - If air leaks in around the nose, re-adjust the noseclip to eliminate leakage. Repeat the above fit check.
 - If air leaks in at the edges, work the straps back along the sides of the head to eliminate leakage.

Storage

Store in a cool dry place, away from frost and light, and in their original packing (-20 to +30 °C, 10 – 80 % relative humidity). Masks last approx. 3-5 years in these conditions.

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GB

Declaration of Conformity (DOC)

JINHUA GIME Safety Protective Product Co.,LTD hereby declares that this product Conforms to the (EU)2016/425 directive.

Click on the following link to read the full text of the EU declaration of conformity:

www.conrad.com/downloads

Enter the product item number in the search box. You can then download the EU

declaration of conformity in the available languages.

General Information

Manufacturer.....JINHUA GIME Safety Protective Product Co.,LTD

Conformity.....EN 149:2001+A1:2009

Regulation(EU)2016/425

Notify Body.....BSI Group

The Netherlands B.V., John M. Keynesplein 9,

1066 EP Amsterdam

Identification number:2797

Manufacturer: Jinhua GIME Safety Protective Product Co., Ltd.
Address : No.178.Yi Village,Bailongqiao Town,
Jinhua City Zhejiang Province,China.
Postal Code : 321025
Tel : 86-0579-82210747 Lot No:20220511
Fax : 86-0579-82210462 Expiry Date:2025/05/11
Email: jimie@gime-cn.cn
www.gime-cn.cn

distributed by
Conrad Electronic SE
Klaus-Conrad-Str. 1
D-92240 Hirschau