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Best.-Nr. 2575496

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ⓑ Operating Instructions

FFP3 Particulate Respirator with Valve 10 pcs

Item No. 2575496

Compliance: EN149:2001+ A1:2009

These single-use particle filter half masks comply with the statutory national and European requirements. All company names and product names are trademarks of their respective owners. All rights reserved.

Warning

- Contact with skin may cause allergic reactions in people with sensitive skin. In this event, leave area of risk, remove mask, and seek medical advice.
- These operating instructions cannot replace a risk assessment or purchase advice. The user has to be trained and instructed in wearing the mask.
- When choosing and using the correct respiratory protection, a detailed analysis is required of the workplace and the hazardous substances used there.
- These respirators may not be used if the concentration type, and properties of contaminants in the ambient atmosphere are unknown or at dangerous levels.
- Respirators should be disposed if damaged, if the breathing resistance becomes high due to clogging or at the end of a single shift (up to 8 hours).
- Only remove the mask from packaging just before you are going to use it.
- The mask must be chosen according to the type of contaminant, its concentration and the protection factors as required by national laws.
- This mask must only be used once (no maintenance).
- Do not modify the mask and check for damage before using.
- Use the mask in well ventilated areas, where oxygen concentration is not less than 19.5%.
- Immediately leave the area if you feel dizzy, faint, or experience breathing difficulties.
- This mask DOES NOT PROTECT the user against gases or vapors and solvents from spray painting.
- Failure to observe these instructions may have adverse effects on the health of the user and expose them to serious and potentially life threatening risks.
- Do not use with facial hair, jewelry or eyewear that may inhibit contact between the face and the product thus preventing a good seal or application of the face mask.

Protection Class

- FFP3: Filtering efficiency (99%), total inward leak (<1%), (APF) Allocated Protection Factor of 10. Applications e.g. Dust, solid and liquid based aerosols.

Instructions for Use

1. Hold the mask in your hand, with nose bar facing towards the top and straps hanging down. Separate top and bottom panels to form a cup shape.
2. Make sure both panels are fully unfolded.
3. Cup respirator in one hand with open side towards face. Take both straps in other hand. Hold respirator under chin, with nosepiece up, and pull straps over your head.
4. Locate the upper strap across the crown of the head and the lower strap below the ears. Straps must not be twisted. Adjust top and bottom panels for a comfortable fit, ensuring panels and tab are not folded in.
5. Using two hands mold the nose bar around the nose and seal it off completely
6. Check mask fit to ensure a good seal.
 - Cup both hands and cover the exhalation valve without disturbing mask fit, then INHALE sharply without pushing on the mask.
 - If air leaks around the nose, re-adjust the noseclip to eliminate leakage. Repeat the above fit check.
 - If air leaks at the respirator edges, work the straps back along the sides of the head to eliminate leakage.

Storage

Store in a cool dry place, away from frost and light, and in their original packing (-20 to +30 °C, 10 – 80 % relative humidity). Masks last approx. 3-5years in these conditions.

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HerstellerJINHUA GIME Safety Protective Product Co.,LTD
The Netherlands B.V.,John M. Keynesplein 9,
JINHUA GIME Safety Protective Product Co.,LTD hereby declares that this product
Manufacturer.....JINHUA GIME Safety Protective Product Co.,LTD