**beurer** FT 15/1   CE 0483   DEUTSCH

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**beurer** FT 15/1   CE 0483   ENGLISH

**Signs and symbols**  
The following symbols are used in these instructions for use, on the packaging and on the type label for the device accessories:

  CE 0483   ENGLISH

**Included in delivery**

- Thermometer
- These instructions for use
- 1 x 1.5V LR 41 battery

**Important safety information – retain for later use.**  
The key to making the most of your Beurer thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.

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**Measuring methods:**  
**Temperature measurement in the anus (rectal)**  
This method of measurement is most reliable and most accurate. Guide the tip of the measuring probe carefully 2-3 cm into the anus. The measuring time with this thermometer at this site is only about 10 seconds. The end of the measuring time is indicated by a signal tone.

**Measurement in the mouth cavity (oral)**  
For this purpose, guide the tip of the thermometer carefully into one of the heat pouches under the tongue. The measuring time with this thermometer at this site is only about 10 seconds.

**Measurement of temperature in the armpit (axillary)**  
This method of measurement is relatively inaccurate. Therefore, it cannot be recommended from a medical point of view.

**Mode of measurement**

Recommended time required	Mean divergence to the rectal method
In the anus	until the acoustic signal
In the oral cavity	until the acoustic signal
In the armpit	until the acoustic signal

**Use**  
Press the On/Off switch to turn the unit on. An acoustic signal announces that it is switched on. The thermometer will carry out a self-test for about 2 seconds. During the self-test all segments of the display will be visible. Then "Lo °C (Lo °F)" or the value of the last measurement will appear in the display. Then the measuring time begins. A short acoustic signal will appear. Then the flashing symbol "C" appears and the symbol "W. Lo °C" is shown in the display screen. The thermometer is now ready for use. During the measurement, the current temperature is displayed in the display. Above the display, a light will blink, when a temperature plateau has been reached. An acoustic signal (10 long beeps) sounds, the "C" ("F") symbol stops flashing and the measured temperature is shown in the display (89.9°F). Higher values are displayed directly, e.g. 32.1°C (89.8°F). If temperatures over 43.0°C (109.4°F) are measured, a lever alarm (30 short beeps in 10 seconds) sounds. If temperatures over 43.0°C (109.4°F) are measured, a signal tone (10 long beeps) is sent. The measurement time is very quick compared to conventional thermometers and is only 10 seconds for the rectal measurement. More accurate measurements are possible by extending the measuring time. If the measuring time is prolonged, a more accurate measurement time should be maintained in spite of the acoustic signal. To prolong the life of the battery, please switch off the thermometer by briefly pressing the On/Off button. Otherwise the thermometer will automatically switch off after 10 minutes.

**Notes on handling batteries**  
If you are using the battery, please clean with battery fluid, flush out the affected areas with water and seek medical assistance.

- **Choking hazard** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- **Observe the disposal and recycling signs.**
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- **Protect the batteries from excessive heat.**
- **Risk of explosion** Never throw batteries into a fire.
- **Do not charge or short-circuit batteries.**
- **Do not use the battery compartment cover as a long period.** Take the batteries out of the battery compartment.
- **Do not use rechargeable batteries.**
- **Do not disassemble, split or crush the batteries.**

**! Notes on handling batteries**  
If you are using the battery, please clean with battery fluid, flush out the affected areas with water and seek medical assistance.

**! Choking hazard** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.

• **Observe the disposal and recycling signs.**

• If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.

• **Protect the batteries from excessive heat.**

• **Risk of explosion** Never throw batteries into a fire.

• **Do not charge or short-circuit batteries.**

• **Do not use the battery compartment cover as a long time.** Take the batteries out of the battery compartment.

• **Do not use rechargeable batteries.**

• **Do not disassemble, split or crush the batteries.**

**Cleaning/Disinfection**  
The device must be cleaned and disinfected before and after each use. The thermometer is not suitable for autoclave, steam or disinfection. It can be dipped into water or disinfected without any difficulties.

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