ACE XS Lite

Breathalyzer

Operating instructions – EN

Atemalkoholtester

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Alcotest

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Alkohol Tester

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Test alcolico

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Alcoholímetro

Manual de instrucciones – ES

www.ace-technik.com

Breathalyzer ACE XS Lite

Operating instructions – EN

Disclaimer

The information in these operating instructions has been compiled by ACE Handels- und Entwicklungs GmbH (ACE) with the utmost care. We assume that all information is correct. Nevertheless, we cannot accept any liability for any consequences arising from the use of this information. ACE is not liable for any costs or damages that may arise from the use of this manual. The contents of this manual are subject to change at any time without prior notice.

Parts and Accessories

Please check the entire contents for all parts and accessories. If anything is missing, please contact info@ace-technik.com, and we will offer you a replacement.

The scope of delivery includes

- 1x ACE XS Lite breathalyzer
- 5 mouthpieces
- 1 storage box with key ring
- instruction manual

Please ensure that all the items listed are present before using the breathalyzer.

Introduction

Congratulations on your purchase of the ACE XS Lite breathalyzer and thank you! The ACE XS Lite is a compact, keychain-sized breathalyzer specifically designed to provide reliable and straightforward breath alcohol measurements for everyday use.

As a lightweight and practical version of the ACE XS, the XS Lite focuses on the essential functions that allow you to quickly assess your blood alcohol concentration (BAC). The intuitive operation and clear display make it an ideal companion for private use.

To ensure accurate and consistent results, please use the ACE XS Lite only as instructed in this manual. The ACE XS Lite offers you a compact solution for on the go without sacrificing accuracy and reliability.

Important instructions for using the ACE XS Lite breathalyzer

- A)** Wait at least 15 minutes after the last sip of alcohol until performing the alcohol test. Do not eat, drink or smoke during this time.
- B)** Do not open the external hardware.
- C)** Do not store the breathalyzer in a cold, hot, damp or poorly ventilated place.
- D)** Keep the breathalyzer out of the reach of children.
- E)** After a longer storage period, carry out three „run-in tests“ before carrying out the actual alcohol test.
- F)** Avoid dropping the breathalyzer.
- G)** Replace the battery if its capacity is low. A reduced battery capacity can lead to incorrect test results.
- H)** Do not carry out a test in noisy environments, as sound waves can interfere with the sensor. Also avoid draughty or poorly ventilated areas.
- I)** After a test result of 1.00 per mille or higher, the sensor requires a longer recovery time. In this case, wait at least three minutes until the next test.
- J)** Avoid testing in very cold, hot or smoky environments with poor ventilation.
- K)** Store the breathalyzer in a place where it is protected from dust and liquids.

- L)** Do not blow too hard into the device, as this can damage the sensor, especially if a lot of saliva gets into the device when blowing.
- M)** Do not store the device near alcohol-based agents such as disinfectants or solvents and do not use them to clean the device.

Preparation for the test

1. Open the battery compartment cover on the top of the device using the notch. Insert the AAA battery (alkaline, 1.5 V), included with delivery, into the battery compartment. Make sure that the polarity (+/-) is correct when inserting the battery and when replacing it later.
2. Insert a mouthpiece into the opening on the right-hand side of the device. Make sure that the mouthpiece is firmly in place. Use a new mouthpiece for each test subject to ensure proper hygiene and maintain measurement accuracy, as a used mouthpiece may contain residual alcohol inside.

Carrying out the test

Press the power button for about 1 second until the device beeps and the ACE logo appears on the display. The device goes through a short warm-up phase, which is indicated by an hourglass on the display. As soon as the device is ready for use, a pictogram of a person blowing appears on the display, accompanied by an acoustic signal.

Note

The device switches off automatically after about 10 seconds without performing any further tests. Before switching off, a beep sounds and „Power Off“ is shown on the display. Another beep is then emitted and the device switches off completely. You can also switch off the breathalyzer manually by pressing and holding the start button for 2-3 seconds. Shortly before switching off, you will hear a beep and „Power Off“ will appear on the display at the same time.

ACE recommends

- Always use a new mouthpiece for each person performing the test! ACE accepts no responsibility for possible damage caused by the use of one mouthpiece by several persons or for several tests.

- Only use hygienically individually packaged mouthpieces from ACE. The test device can become contaminated if shared by several people.
- Wear gloves when removing mouthpieces from the device after a test, especially if different people have carried out the test. ACE Technik (www.ace-technik.com) offers suitable gloves for this purpose.

Important notes

- If you have eaten, drunk alcohol, or smoked, wait at least 15 minutes before taking a measurement.
- This breathalyzer is a high-quality and sensitive measuring device. To maintain the measuring accuracy of the electrochemical sensor, the device should be used regularly. If the device is not used for a long period of time, the sensor may dry out. It is recommended to use the breathalyzer at least twice a month for alcohol testing, even if you have not consumed alcohol.
- For measured values below 0.07‰ (or 0.007% BAC or 0.035 mg/L - depending on the setting), the test result is displayed as 0.00 (‰ or mg/L or % BAC - depending on the setting). This is because other substances may be present in the breath at low alcohol levels that could influence the result of the breath sample.

Only the use of original ACE mouthpieces to

- ensure the accuracy of the measurement results.
- guarantee zero cross contamination.
- secure the full device warranty.

Warning from ACE

Even if the device indicates an alcohol level in the „safe“ range, this does not mean that you can safely drive a vehicle or carry out other activities that could be dangerous under the influence of alcohol after consuming alcohol.

Warnings on the display

- **Blow Error** - If the breath volume is insufficient, the OLED display shows “Blow” and an “Error” symbol. In this case, briefly press the power button to restart the measurement.
- **Low bat** - The battery capacity is low. The OLED display shows a “Low

- Battery" symbol. Please replace the battery with an AAA alkaline battery.
- **CAL** - After more than 1,000 tests, a calibration message is displayed. Please have the device calibrated at the service center.
 - **Err1** - Displayed when the sensor needs to be checked. Please contact the service center.
 - **Err2** - Displayed if the breath sampling needs to be checked. Please contact the Service Center.
 - **Err3** - Displayed if the operating temperature range (5-40 °C) is exceeded. Please use the device within the recommended temperature range

Calibration

ACE recommends calibrating the integrated sensor approximately every 12 months or after 500 tests (whichever is sooner). Although calibration is optional, the accuracy of the sensor will decrease without regular calibration and the sensor warranty will be voided. You can send the device for calibration to the service address given at www.ace-technik.com.

We recommend sending the device as a parcel in order to provide proof of delivery or a follow-up request in the event of non-delivery. Please note that the procedures described may change due to product modifications without this being updated in the operating instructions. Such changes do not entitle the user to a complaint, return or price reduction. It is the user's responsibility to recognize deviations between the operating instructions and the actual operation of the breathalyzer and to act accordingly.

Notes on the batteries

The device referred to in these operating instructions is supplied with 1 AAA alkaline battery. The battery are not rechargeable. Do not use rechargeable batteries, as this may damage the device. Empty batteries must not be disposed of with household waste as they contain potentially harmful substances that can damage the environment and your health.

Please return the batteries to retailers or the recycling centers in your municipality. The return is free of charge and required by law. You should tape the terminals of lithium batteries before disposing of them.

Battery change

To replace the battery, slide the battery compartment cover on the back of the device downwards and replace the discharged batteries with new ones. Make sure that the polarity is correct: the negative pole of the battery must be in contact with the springs in the battery compartment. Then replace the battery compartment cover on the back of the device and press it gently until it clicks into place.

Note on battery recycling

Recycling batteries is a simple measure to protect the environment and community health. Please do not dispose of old batteries and rechargeable batteries in household waste. They can be recycled, allowing valuable raw materials such as iron, zinc and nickel to be recovered.

Technical data

- Sensor type - Fuel Cell sensor
- Battery - 1x alkaline battery (AAA size)
- Battery life: approximately 150 uses per battery
- Weight - 60g including battery
- Dimensions - 67,8 mm (Height) x 51,5 mm (Width) x 20,4 (Thickness)
- Measurement Results:
OLED: 0.00 - 4.00‰, 0.00 - 2.00mg/L, 0.00 - 0.40%
- Buzzer: Silent
- Warming up time: 5 seconds
- Accuracy: +/-10% at 0.50‰ / 0.050‰BAC / 0.25mg/L
- Operating Temperature 5°C - 40°C
- Storage Temperature 5°C - 60°C
- Mouthpiece: MP-Ki Black
- Automatic Power OFF: Automatically turns off after 10 seconds of inactivity
- Calibration Period: Every 1.000 tests
- Display Time of Measurement Results: 10 seconds
- Button operations While Display Measurement Results:
One press (less than 3 seconds): Retake measurement
Long press (more than 3 seconds): Power OFF
- Features: Calibration request notification based on the number of measurements

Warranty and liability

ACE Handels- und Entwicklungs GmbH (ACE) guarantees the product to which this warranty and liability declaration is attached - in all versions and designations - against defects in material and workmanship for a period of two years from the date of purchase and a one-year warranty on the sensor. This warranty covers the electronics and the external hardware but refers only to the sensor without sampling components. Parts that are inevitably destroyed during use (e.g. fuses) are excluded from the warranty. The warranty begins on the date of delivery or first use (whichever is shorter), unless otherwise specified in the literature accompanying the ACE product.

WARRANTY AND LIMITATION OF LIABILITY

Product restriction

ACE disclaims all other warranties, express or implied, including the implied warranties of merchantability and fitness for a particular purpose. In particular, the warranty does not apply to calibration. Calibration is mandatory after sensor replacement. If a sensor replacement and/or calibration is necessary, the user may receive a cost estimate. If the user rejects the cost estimate, ACE will return the product unrepaired/uncalibrated and mark it accordingly. If the product does not fulfill this warranty, the purchaser's claims and ACE's obligations shall be limited to the replacement or repair of the goods concerned or a refund of the purchase price for the goods complained about, at its own discretion.

Application restriction

In no event shall ACE be liable for any additional, incidental or consequential damages, including, but not limited to, lost profits and loss of use, arising out of the sale, manufacture or use of the products sold under this disclaimer, whether such claims arise in contract, negligence or other tortious action.

In particular, the product to which this warranty and liability statement is attached, regardless of the version and designation, cannot be used in a legally binding manner to determine the fitness of a person to drive a vehicle or to carry out an action which, under the influence of alcohol, results in danger to himself or others. Careful inspection of all products upon receipt for damage, calibration suitable for the purchaser's particular application and proper use, repair and maintenance in accordance with the instructions in the product literature supplied by ACE are mandatory requirements for this warranty. Repair or maintenance by unqualified personnel will void the warranty, as will the use of non-approved consumables or replacement parts. As with all technically sophisticated products, it is a fundamental and mandatory requirement of ACE' warranty that all users of these products are familiar with the use, capabilities and limitations of the product as described in the applicable product literature.

Consultation restriction

The Buyer acknowledges that they have verified the intended use and suitability of the purchased goods for any purpose under its sole responsibility. It is expressly agreed between the parties that any technical or other advice given by ACE regarding the use of the goods or services is given free of charge and at the Buyer's sole risk. Therefore, ACE shall have no obligation or liability whatsoever arising from the recommendations or the results thereof.

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ACE Handels- & Entwicklungs GmbH